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Safety instructions

1. Do not use this product on newborns toddler, children, person who cannot express their consent and preeclampsia patients. We recommend consulting your doctor before using the product if you are pregnant.
2. The measurements determined by you are only intended for your information - they do not replace a medical examination. Discuss the measurements with your doctor. Do not self-diagnose or self-treat based on the measurement results without consulting your attending physician. Do not start new medication without consulting a doctor and do not modify the type and / or dosage of current medications.
3. The product may not be used in conjunction with a high-frequency surgical unit.
4. Cardiovascular system diseases may result in incorrect measurements and/or impairments to the device's measurement accuracy. This is also the case when it comes to low blood pressure, diabetes, circulatory problems and dysrhythmia, as well as chills and shaking. Patients in shock, people with electrical implants such as a cardiac pacemaker.
5. Only use the product on individuals who have the upper arm range stated for the product.
6. Ensure that the limb is not functionally impaired during inflation.
7. Blood circulation should not be restricted by the blood pressure measurement for an unnecessary amount of time. Remove the cuff from the arm if the device malfunctions.
8. Avoid mechanically restricting, compressing or kinking the cuff tubing.
9. Avoid sustained pressure to the cuff, as well as frequent measurements. The resulting curtailing of the blood flow can lead to injuries.
10. Ensure that the cuff is not placed on an arm whose arteries or veins are undergoing medical treatment, i.e. intravascular access or intravascular therapy or an arteriovenous (A-V) shunt.
11. Do not place the cuff on individuals who have had a mastectomy.
12. Do not place the cuff over wounds as this could lead to further injuries.
13. Only place the cuff on the upper arm. Do not place the cuff on other body parts.
Attention! The device may only be used in conjunction with the original cuff provided.
14. Do not use rechargeable batteries.
15. If no button is pressed, the automatic shutoff will switch off the blood pressure monitor within 1 minute to save the batteries.
16. Protect the device from impacts, moisture, dirt, severe temperature fluctuations and direct sunlight.
17. Do not drop the device.
18. The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device incorrect measurement result.
19. Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is

Safety instructions

- necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
20. The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
 21. Failure to comply with the above can impair the performance of the device.
 22. Do not use the device in the vicinity of strong electromagnetic fields, keep it away from radio equipment and mobile telephones.
 23. This product is not intended for commercial use. It is designed for household use only.
 24. Place this product on a horizontal and even surface only.
 25. Read this instruction manual carefully before first use. It contains important safety information as well as instructions for the use and care of the device. Keep these instructions for future reference and pass them on with the product.
 26. To avoid dangerous situations, do not use this product for any purposes other than those described in the instructions. Improper use is dangerous and voids the warranty.
 27. **Risk of injury!** Do not use this product if it is damaged or has fallen to the ground.
 28. **Caution!** There is a danger of explosion from improper replacement of the batteries. Replace the batteries only with the same or equivalent type of batteries.
 29. Make sure the batteries are correctly inserted. Observe the polarity (+) positive / (-) negative on the batteries and the housing. Improperly inserted batteries can lead to leakage or, in extreme cases, a fire or an explosion.
 30. The connections must not be short-circuited.
 31. Never expose batteries to excessive heat (e.g. bright sun, fire), and never throw them into fire. The batteries could explode.
 32. Remove used batteries. Remove the batteries if you know that the product will not be used for a longer period of time. Otherwise, the batteries could leak and cause damage.
 33. Non-rechargeable batteries must not be recharged.
 34. If the batteries leak, remove them with a cloth and dispose of them accordingly. Prevent battery liquid from coming into contact with skin and eyes. If you do get battery liquid in your eyes, flush them thoroughly with lots of water and consult a physician immediately. If battery liquid comes into contact with your skin, wash the affected area with lots of water and soap.
 35. The possible incorrect measurements of the equipment may be caused by external interference such as transportation acceleration.
 36. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the blood pressure monitor, including main parts specified by the manufacturer. Otherwise, degradation

Safety instructions

- of the performance of this device could result.
37. The "irregular heartbeat" function does not replace a cardiac examination, but may help to detect potential heart rate irregularities at an early stage. Always consult your physician to determine what will be suitable for you.
 38. The "irregular heartbeat" function is not designed for diagnosing or treating an arrhythmic disorder. Arrhythmia can only be ascertained by a licensed physician.
 39. The "WHO BLOOD PRESSURE CLASSIFICATION" chart is not intended to replace a medical diagnosis. This chart is only for reference for different classifications of blood pressure.
 40. If you notice abnormal or suspicious variations in blood pressure measurements, consult your physician immediately.
 41. Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
 42. This product contains small parts that may present a choking hazard to children. Keep the unit and all parts out of reach of children.
 43. The tubing presents a strangulation hazard. Keep this product away from children and those who require close supervision, e.g. people with mental disorders.
 44. Do not drape tube around neck. This presents a strangulation hazard.
 45. Remove any kind of arm jewellery or the like before taking a measurement. This could cause bruises.
 46. Do not place the arm cuff over heavy clothing (e.g. a jacket or sweater sleeve) as the blood pressure monitor will not be able to take a proper measurement and there is an elevated danger of acquiring hematoma or skin marks during the course of the measurement.
 47. When applying the cuff, make sure there are no wrinkles in the cuff as this could cause bruises.
 48. Blood pressure measurements can lead to temporary marks on the skin at the site of the cuff placement. This is especially the case in high repetition rates, in hypertonic patients and in patients with weak heart rates. In rare cases a mark may persist for couple of days. Please contact your physician about these specific risks of cuff pressure in your specific case.

This device conforms to Regulation (EU) 2017/745 concerning medical devices. This is made evident by the CE mark of conformity accompanied by the reference number of the designated authority. This device complies with:

- EN ISO 81060 standard relating to non-invasive sphygmomanometers Part 1: Requirements and test methods for non-automated measurement types and EN 1060 standard relating to non-invasive sphygmomanometers Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems.
- EN 60601 standard relating to medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance and essential

Safety instructions

performance – Collateral standard:
Electromagnetic compatibility –
Requirements and tests.

- ISO 81060-2:2018+A1:2020
standard relating to non-Invasive
sphygmomanometers - Part 2: Clinical
validation of automated measurement
type.
- IEC 80601-2-30:2018 standard relating
to medical electrical equipment - Part
2-30: Particular requirements for the
basic safety and essential performance
of automated type non-invasive
sphygmomanometers.

Congratulations!

Thank you for your purchase of a **KOENIC** product. Please read this manual carefully and keep it for future reference.

Intended use

This upper arm blood pressure monitor is used for the non-invasive measurement and monitoring of arterial blood pressure values of adult humans. This product is not intended for commercial use. It is designed for household use only. The Imtron GmbH assumes no liability for damage to the product, for property damage or for personal injury due to careless or improper usage of the product, or usage of the product which does not meet the manufacturer's specified purpose.

Before first time use

Remove the product and its accessories carefully from the original packaging. It is recommended to keep the original packaging for storage. If you wish to dispose of the original packaging, please observe applicable legal provisions. Should you have any questions regarding proper disposal, contact your local waste management center. Inspect the delivery contents for completeness and damages. Should the delivery contents be incomplete or damaged, contact your sales outlet immediately.

After unpacking, please refer to chapter **Cleaning and care**.

Content

1 x Blood pressure monitor with upper arm cuff
1 x Storage bag
1 x User manual

Controls and components

- A** Display
 - 1** Average of previous measurements
 - 2** Battery low indicator
 - 3** Date
 - 4** Time
 - 5** User memory
 - 6** Inflating
 - 7** Release pressure
 - 8** Indicator: severe hypertension
 - 9** Indicator: moderate hypertension
 - 10** Indicator: mild hypertension
 - 11** Indicator: high normal blood pressure
 - 12** Indicator: normal blood pressure
 - 13** Indicator: optimal blood pressure
 - 14** Pulse rate
 - 15** Indicator: Irregular heartbeat
 - 16** Diastolic pressure
 - 17** Memory number
 - 18** Systolic pressure
- B** Storage bag
- C** Cuff
- D** Cuff hose
- E** Mark
- F** Scale for the classification of measurement results
- G** Socket for cuff plug
- H** Cuff plug
- I** Battery compartment with lid
- J** Memory button **M**
- K** Button **▲**
- L** Button **▼**
- M** **Start/Stop** button **⏹**
- N** Power supply connector

English

Preparation

Before first use, the cuff must be connected to the device and batteries must be inserted. Alternatively, you can operate the device with a 6V DC power supply.

Insert or change the batteries or connect the power adapter

- Fig. 1
1. Open the battery compartment by removing the battery cover.
 2. Remove old batteries if necessary. Insert 4 LR03 / AAA batteries. Observe the correct polarity (+/-) according to the imprints on the batteries and in the battery compartment.
 3. Close the battery compartment.

Note:

- Do not use rechargeable batteries. The device may not work properly with rechargeable batteries because of the lower voltage.
- If the error message **E6** is shown in the display, the batteries are empty and must be replaced

- Fig. 2
2. Alternatively, you can connect a suitable power supply to the DC jack, with output 6V DC, at least 600 mA and DC plug 5.5 mm / 2.1 mm.

Setting the region, date and time

- Fig. 3
1. Immediately after inserting the batteries **EU** flashes on the display. Select the region **EU** (Europe) or **US** (USA) with the buttons **▲** / **▼** and confirm your selection by pressing **M**.
 2. The symbol for the year now flashes in the display. Select the year with the **▲** / **▼** buttons and confirm your selection by pressing **M**.
 3. The symbol for the month now flashes in the display. Select the month with the **▲** / **▼** buttons and confirm your selection by pressing **M**.
 4. The symbol for the day will now flash in the display. Select the day with the **▲** / **▼** buttons and confirm your selection by pressing **M**.
 5. The symbol for the hour now flashes in the display. Select the hours with the **▲** / **▼** buttons and confirm your selection by pressing **M**.
 6. The symbol for the minute display now flashes in the display. Select the minutes with the **▲** / **▼** buttons and confirm your selection by pressing the **M** button. The settings are now complete.

Measure blood pressure

- To ensure comparability of the values, always measure your blood pressure at the same time of day. Rest for about 5 minutes before each measurement!
- If you want to take multiple measurements on one person, wait 5 minutes between each measurement.
- Do not eat, drink, smoke, or exercise for at least 30 minutes before the measurement.
- Repeat the measurement in case of doubtful values.
- Let the device acclimatize to room temperature before measuring.

Fasten the cuff

Warning

The device shall only be operated with the supplied cuff. The cuff is suitable for an upper arm circumference of 22 to 36 cm.

- Fig. 4
- Insert the cuff plug into the socket at the blood pressure monitor.
 - Put the cuff on the bared left upper arm. The circulation of the arm must not be restricted by too tight clothing or the like.
 - The cuff should be placed on the upper arm so that the lower edge is 2-3 cm above the crease of the arm and above the artery. The sleeve tube points towards the hand.
 - Now put the free end of the cuff tight, but not too tight around the arm and close the Velcro fastener. The cuff should be so tight that two fingers fit under the cuff.

Correct posture

Note:

Be calm and do not move during the measurement and do not speak to prevent a false measurement result.

- Fig. 5
- You can take the measurement while sitting or lying down. Always make sure that the cuff is at heart level.
 - To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.

Select user memory

- Fig. 6
- Press any button. Select the desired user by pressing the function buttons **Λ** / **∇**.
 - You have 4 memories with 120 memory spaces each to save the measurement results of 4 different people separately.
 - Confirm your selection with the **Start/Stop** button or the **M** button. Your selection will then be saved automatically.

Blood pressure measurement

- Fig. 7
- Start the blood pressure monitor with the **Start/Stop** button **⓪**. After the display check showed all segments the last stored measurement result is displayed. If no measurement is saved "0" is shown. The cuff is automatically inflated. The cuff air pressure is slowly released. If there is already a tendency to detect high blood pressure, pumping is continued and the cuff pressure is increased again. As soon as a pulse is detected, the symbol Pulse is displayed. The results Systolic pressure, Diastolic pressure and Pulse are displayed. You can cancel the measurement at any time by pressing the **Start/Stop** button **⓪**.

Note:

If the measurement was not performed properly, a corresponding error code **E ...** is displayed. Please note the clause **Error messages and troubleshooting** Repeat the measurement in these cases.

The measurement result is automatically saved. Turn the blood pressure monitor off by pressing the **start/stop** button again. To switch off, press the **Start/Stop** button **⓪**.

Note:

The unit automatically shuts off after one minute when no button is pressed. Wait at least 5 minutes before the next measurement!

English

Evaluate readings Irregular heartbeat

This device can identify possible heart rhythm disturbances during the measurement and may indicate this after measurement with the symbol . This can be an indicator of an irregular heartbeat.

Irregular heartbeat is a disease in which the heart rhythm is abnormal due to errors in the bioelectrical system that controls the heartbeat. Symptoms (missed or premature heart beats, slow or too fast heart rate) may occur from heart disease, age, physical predisposition, stimulants in excess, stress or lack of sleep. Irregular heartbeat can only be detected by being examined by your doctor.

Repeat the measurement when the symbol appears on the display after the measurement. Please ensure that you rest for 5 minutes and do not speak or move during the measurement.

Warning

If the symbol appears more frequently, consult your doctor. Self-diagnosis and treatment based on the measurement results can be dangerous. Be sure to follow the instructions of your doctor.

Classification of the measurement results

According to the guidelines/definitions of the World Health Organization (WHO) and recent findings, the measurement results can be classified and assessed according to the following table:

Blood pressure values	Systole (in mmHg)	Diastole (in mmHg)	Action
Grade 3: severe hypertension	≥ 180	≥ 110	Seek medical attention
Grade 2: moderate hypertension	160-179	100-109	Seek medical attention
Grade 1: mild hypertension	140-159	90-99	Regular monitoring by a doctor
High normal blood pressure	130-139	85-89	Regular monitoring by a doctor
Normal blood pressure	120-129	80-84	Self-monitoring
Optimal blood pressure	< 120	< 80	Self-monitoring

Source: Journal of Hypertension 1999, Vol 17 No. 2

Note:

These default values are for general guidance only, as individual blood pressure differs for different people and ages, and so on. It is important that you consult your doctor at regular intervals. Your doctor will tell you your individual values for a normal blood pressure and the level at which the level of blood pressure is considered dangerous.

The bar graph in the display and the scale on the device indicate in which area the blood pressure is located.

If the values of systole and diastole are in two different ranges (eg, systole in the "high normal" and diastole in the "normal" range), the graphic scale on the device will always show you the higher range.

Store, recall and delete measured values

The results of each successful measurement are stored together with the date and time. If there are more than 120 measurement data, the oldest measurement data will be overwritten.

- Press the **▲** / **▼** buttons to select the desired user memory. Confirm your selection with the **Start/Stop** button **⓪** or the **M** button. Press the **M** button to display the **AVG** average of all saved user memory readings. Press the **▲** / **▼** buttons to display the average of the last 7 days of the morning (AM, 5:00-9:00) and evening (PM, 18:00-20:00) measurements. Pressing the **▲** / **▼** button again displays the last individual measured values with date and time.
- Press the **Start/Stop** button **⓪** to turn off the unit.

Note:

The unit shuts off automatically after one minute when no button is pressed.

Delete data manually

1. Press **M** button simultaneously when the device is showing average or previous measurement data of selected user.
2. Press and hold **▲** and **▼** for 5 seconds until **CL** and **00** are shown.
3. All measurement data for the selected user will be deleted.

Cleaning and care**Warning**

Never use solvents and abrasives, hard brushes, metallic or sharp objects for cleaning. Solvents are harmful to health and attack the plastic parts; scouring cleaners and aids scratch the surface.

- Clean the outside of the product with a slightly damp cloth and then dry well.
- Make sure that no water (e.g. at the buttons) gets inside the product. Store the product in a dry, cool place protected from moisture and direct sunlight. Store the product in a place inaccessible to children.

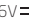
English

Error messages and trouble shooting

If there are any problems during operation, please refer to the following table. If problems cannot be solved as indicated under **Solution**, please contact your retailer.

Problem	Possible causes	Solution
Nothing appears in the display, even when the Start/Stop button  is pressed	• Batteries drained	• Replace with new batteries
	• Batteries wrongly inserted	• Remove batteries and insert again correctly considering the correct polarity
	• When power supply is used: DC plug is not inserted correctly or power supply is not plugged in	• Insert DC plug again and make sure power supply is plugged in
Error code 1 (E1)	• Cuff position is not correct	• Sit comfortably and still. Ensure that the cuff is at the same level as the heart
Error code 2 (E2)	• You moved your arm or body during measurement	• Make sure you remain very still and quiet during the measurement
Error code 3 (E3)	• The cuff position is not fastened properly	• Fasten the cuff correctly
	• The cuff may not be connected	• Check whether hose connection of the cuff is secured to the unit properly
Error code 4 (E4)	• The unit does not measure	• If you have a very weak or irregular heart beat, the device may have difficulty in determining your blood pressure
	• There is a measuring Error	• Sit comfortably and still. Fasten the cuff again carefully
Error code 5 (E5)	• Cuff over inflated	• The measurement range is over 300 mmHg. See a doctor as soon as possible.
Error code 6 (E6)	• Low battery	• The battery power is too low to function. Replace the batteries with new ones.
The blood pressure monitor keeps reinflating	• Circuit locked	• Remove and reinsert the batteries and then proceed to take measurement again.

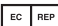
Technical data

Model No.: MD3111 (KBP 2024)	
Measurement Method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measuring range / resolution	Systolic 50-250 mmHg, diastolic 30-200 mmHg / 1 mmHg, pulse 40-180 beats / minute / 1 bpm
Accuracy of the display	Systolic and diastolic ± 3 mmHg, pulse $\pm 5\%$ of the indicated value
Uncertainty	Max. permissible standard deviation according to clinical examination: systolic 8 mmHg / diastolic 8 mmHg
Storage	4 x 120 memory slots
Dimensions	99 x 136 x 45 mm
Cuff	Suitable for an arm circumference of 22 to 36 cm
Perm. Operating conditions	+5 °C to +40 °C, 15% to 90% relative humidity (non-condensing)
Perm. Storage conditions	-25 °C to +70 °C, <90% relative humidity, 700-1060 hPa ambient pressure (For device only.) Battery: Please refer to manufacturer's specification
Power supply	4 x 1.5 V LR03 / AAA batteries
Expected product life	5 years
Battery life	Approx. 200 measurements, depending on the level of blood pressure or inflation pressure
Equipment	Instructions for use, 4 x 1.5 V LR03 / AAA batteries, storage bag
Classification	Internal supply, IP21, no AP or APG, continuous operation, application part type BF
Power supply * (not included)	
Input	100-240 V, 50-60 Hz
Output	6V  , 600 mA (lifetime 5000 times), DC hollow plug: 5.5 mm / 2.1 mm 

* Please noted that a medical grade AC adapter shall be used.

English

Key to Symbols

Symbols	Function / Meaning
Key to Symbols	Application part Type BF 
	Class II equipment symbol 
	Manufacturer
	Type BF: Device, cuff and tubing are designed to provide special protection against electrical shocks.
	EC Directive Medical Device Label
	Caution
	Authorized Representative in the European Community
	Refer to instruction manual / booklet
	Keep dry
	This item is a medical device

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Appendix I

Guidance and manufacture's declaration - electromagnetic immunity			
The Sphygmomanometer MD3111 (KBP 2024) is intended for use in the electromagnetic environment specified below. The customer of the user of Sphygmomanometer MD3111 (KBP 2024) should assure that it is used in such an environment.			
Immunity test	ICE 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Rated power frequency magnetic fields IEC 61000-4-8	50 Hz: 30 A/m 60 Hz: 30 A/m	50 Hz: 30 A/m 60 Hz: 30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Appendix

Appendix II

Guidance and manufacture's declaration - electromagnetic immunity			
The Sphygmomanometer MD3111 (KBP 2024) is intended for use in the electromagnetic environment specified below. The customer of the user of Sphygmomanometer MD3111 (KBP 2024) should assure that it is used in such an environment.			
Immunity test	ICE 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted disturbances induced by RF fields IEC 61000-4-6	3 Vrms ^a 0.15 MHz - 80 MHz 6 Vrms ^a in ISM and amateur radio bands between 0.15 MHz and 80 MHz ^b 80% AM at 1 kHz	3 Vrms ^a 0.15 MHz - 80 MHz 6 Vrms ^a in ISM and amateur radio bands between 0.15 MHz and 80 MHz ^b 80% AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer to any part of the Sphygmomanometer MD3111 (KBP 2024), including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{6}{E} \right] \sqrt{P}$
Radiated RF EM fields IEC 61000-4-3	10 V/m ^c 80 MHz - 2.7 GHz 80% AM at 1 kHz	10 V/m ^c 80 MHz - 2.7 GHz 80% AM at 1 kHz	where P is the maximum output power rating of the transmitter in Watts (W), d is the minimum recommended separation distance in meters (m), and E is the immunity test level in V/m. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
^a r.m.s., before modulation is applied. ^b The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz. ^c Before modulation is applied.			

Appendix III

Recommended separation distances between portable and mobile RF communications equipment and the Sphygmomanometer MD3111 (KBP 2024)		
The Sphygmomanometer MD3111 (KBP 2024) is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Sphygmomanometer MD3111 (KBP 2024) can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Sphygmomanometer MD3111 (KBP 2024) as recommended below, according to the maximum output power of the communications equipment.		
Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)	
	3 Vrms	10 V/m
0.01	0.200	0.060
0.1	0.632	0.190
1	2.000	0.600
10	6.33	1.90
100	20.0	6.00
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.		

Appendix IV

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced technician for help.

**WARNING**

- To comply with the limits of the Class B digital device, pursuant to Part 15 of the FCC Rules, this device is comply with Class B limits. All peripherals must be shielded and grounded. Operation with non-certified peripherals or non-shielded cables may results in interference to radio or reception.
- Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

Appendix

Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.

Grandway Technology (Shenzhen) Limited
No. 5, the Second Industrial Zone, Zhukeng Community,
Longtian Street, Pingshan District, Shenzhen, 518118
The People's Republic of China

MedNet EC-REP GmbH
Borkstrasse 10, 48163 Munster, Germany.