DE	Diagnosewaage Gebrauchsanweisung	2
EN	Diagnostic bathroom scale Instructions for use	12
FR	Pèse-personne impédancemètre Mode d'emploi.....	21
ES	Báscula diagnóstica Instrucciones de uso.....	31
IT	Bilancia diagnostica Istruzioni per l'uso.....	41

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Contents

1. Included in delivery.....	12	7. Usage	17
2. Signs and symbols	13	8. Cleaning and maintenance.....	19
3. Intended use.....	14	9. What if there are problems?	20
4. Warnings and safety notes	14	10. Disposal.....	20
5. Device description.....	16	11. Technical specifications	20
6. Initial use	16	12. Warranty/service.....	20

Getting to know your device

Functions of the device

This digital scale is intended for weighing and providing a diagnosis of your personal fitness data. It is intended for private use.

The scale is equipped with the following functions, which can be used by up to 10 different people:

- Body weight measurement and BMI
- Body fat percentage calculation
- Body water content
- Muscle percentage

The scale also includes the following additional functions:

- Switch between kilograms “kg”, pounds “lbs” and stones “st”
- Automatic switch-off function
- Battery replacement display for low batteries

The measurement principle

This scale works on the principle of B.I.A. (bioelectrical impedance analysis). This involves the calculation of body content in a matter of seconds, using a current that cannot be felt, is completely harmless and does not pose any risk.

When this measurement of the electrical resistance (impedance) is considered alongside constants and/or individual values (age, height, gender, activity level), the body fat percentage and other variables in the body can be calculated. Muscle tissue and water conduct electricity well and therefore have a lower resistance. In contrast, bones and fat tissue have low conductivity as the fat cells and bones are barely able to conduct the current due to very high levels of resistance.

Please note that the values calculated by the diagnostic bathroom scale only represent an approximation of the actual medical analytical values for the body. Only a specialist doctor can precisely establish the body fat, body water, muscle percentage and bone structure using medical methods (e.g. computer tomography).

1. Included in delivery



Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- SR BF 2 diagnostic bathroom scale
- 1 x 3 V CR2032 battery
- These instructions for use

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning notice indicating a risk of injury or damage to health.		
	IMPORTANT Safety note indicating possible damage to the device/accessory.		
	Product information Note on important information		
	Read the instructions		
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive (WEEE)		
	Do not dispose of batteries containing hazardous substances with household waste		
	Separate the packaging elements and dispose of them in accordance with local regulations.		
	Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard		
	This product satisfies the requirements of the applicable European and national directives.		
	The device may not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.		
	Manufacturer		
	United Kingdom Conformity Assessed Mark		
	Separate the product and packaging elements and dispose of them in accordance with local regulations.		
	Importer symbol		
	Do not load the scale beyond 180 kg/396 lb/28 st.		Danger of slipping: do not step on the scale with wet feet.
			

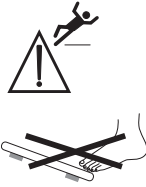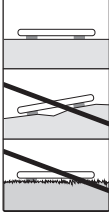	<p>Danger of tipping: position yourself in the centre of the weighing surface.</p>		<p>Place the scale on an even surface. No carpet.</p>
--	--	--	---

3. Intended use

The device is only intended for weighing humans and for recording your personal fitness data. The device is only intended for private use, and not for medical or commercial purposes.

4. Warnings and safety notes

WARNING

- **The scale may not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.**
- Do not use during pregnancy.
- Do not step onto the outer edge of the scale on just one side: danger of tipping!
- Swallowing batteries can be extremely dangerous. Keep the batteries and scale out of the reach of small children. Should a battery be swallowed, seek medical assistance immediately.
- Keep packaging material away from children (risk of suffocation).
- Important: do not step onto the scale with wet feet or if the surface of the scale is damp—danger of slipping!

Safety notes on handling batteries

- Always insert the batteries correctly, taking into account the polarity (+ / -). Keep batteries clean and dry and away from water. Always select the correct battery type.
- Never short-circuit batteries and battery compartment contacts.
- Never charge, forcibly discharge, heat, disassemble, deform, encapsulate or modify batteries.
- Never weld or solder on batteries.
- Never mix batteries of different manufacturers, capacities (new and used), size and type within a device.
- **Risk of explosion!** Failure to comply with the points mentioned above can result in personal injury, overheating, leakage, venting, breakage, explosion or fire.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- **Choking hazard!** Keep batteries out of the reach of children. Seek medical attention immediately if swallowed.
- Never allow children to replace batteries without adult supervision.
- Store batteries away from metal objects in a well-ventilated, dry and cool place.
- Never expose batteries to direct sunlight or rain.
- Remove batteries from the device if it is not going to be used for a long period of time.
- Dispose of discharged batteries immediately and properly. Never dispose of batteries in fire.
- When disposing of batteries, keep batteries with different electrochemical systems separate.

General notes

- Please note that measuring tolerances are possible for technical reasons, as this scale is not calibrated for use in a professional medical context.
- Ages 10 to 100 years and height settings from 100 to 220 cm (3' 3"–7' 3") can be preset. Load capacity: max. 180 kg (396 lb, 28 st). Result in steps of 100g (0.2 lb, 1/4 st). Measurements of body fat, body water and muscle content in steps of 0.1%.

- When supplied to the customer, the scale is set to weigh and measure in “cm” and “kg”. On the rear of the scale, there is a switch where you can select “inch”, “pound” and “stone” (lb, st).
- Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.
- Repairs may only be carried out by Beurer Customer Services or authorised retailers. Before submitting a complaint, please check the batteries first and replace them if necessary.

Storage and maintenance

The accuracy of the measurements and service life of the device depend on its careful handling:

IMPORTANT

- The device should be cleaned from time to time. Do not use any abrasive cleaning products and never submerge the device in water.
- Make sure that no liquids come into contact with the scale. Never submerge the scale in water. Never rinse it in running water.
- Do not place any objects on the scale when it is not in use.
- Do not press the button violently or with pointed objects.
- Do not expose the scale to high temperatures or strong electromagnetic fields (e.g. mobile telephones).
- Protect the device from knocks, damp, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).

General tips

- As far as possible, weigh yourself at the same time of day (ideally in the morning), after having been to the toilet, on an empty stomach and without clothing, in order to obtain results which can be compared.
- Important point regarding the measurement: the calculation of body fat may only be made when bare-foot; the soles of the feet may be lightly dampened where appropriate. Results may be unsatisfactory if the soles of the feet are completely dry or have a large amount of hard skin since conductivity will be impaired.
- Stand up straight and still during the measurement.
- Wait a few hours after any physical exertion to which your body is unaccustomed.
- Wait approx. 15 minutes after getting up so that the water stored in the body can disperse.
- Remember that only the long-term trend is important. Short-term changes in weight within a few days are normally caused by loss of fluids; however, body water plays an important role in terms of well-being.


Limitations

When calculating body fat and the other values, unusual and implausible results may occur in the case of:

- Children under approx. 10 years of age,
- Performance athletes and body-builders,
- Persons with a fever, persons undergoing dialysis, persons with oedema symptoms and persons suffering from osteoporosis,
- Persons taking cardiovascular medicine (affecting the heart and vascular system),
- Persons taking vasodilative or vasoconstrictive medication,
- Persons with significant anatomical abnormalities in their legs compared with their overall body height (leg length significantly shorter or longer).

5. Device description

1. Electrodes
2. Display
3. "Down" button (pushbutton on the front side)
4. "SET" button (pushbutton on the front side)
5. "Up" button (pushbutton on the front side)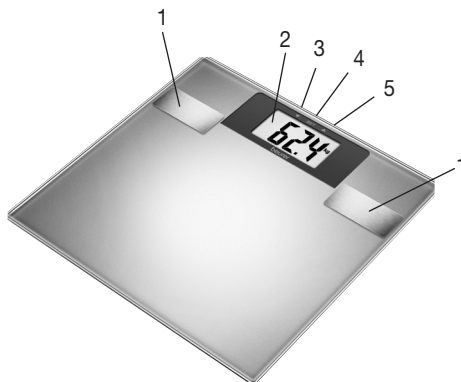

6. Initial use

Batteries

If present, remove the battery insulating strip on the battery compartment lid and remove the protective film from the battery, then insert the battery the correct way round. If the scale does not work, remove the battery completely and reinsert it.

Changing batteries

Your scale is equipped with a "low battery indicator". If you operate the scale with flat batteries, "Lo" will appear on the display and the scale will automatically switch off. In this case, the battery must be replaced (1 x 3V CR2032 battery).

Positioning the scale

Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.


Entering user data

To calculate your body fat percentage and additional body values, you must save the personal user data.

The scale has 10 user memory spaces which allow you and members of your family to save and access personal settings. Switch on the scale (tap the weighing area). Wait until "0.0" appears in the display.

Then press "SET". The first memory space now appears flashing in the display. You can now make the following settings:

Memory space	1 to 10
Height	100 to 220 cm (3' 3" to 7' 3")
Age	10 to 100 years
Gender	Male (♂), female (♀)
Activity level	1 to 5

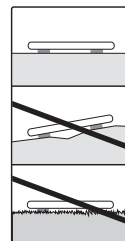You can set the relevant values with a long or brief press of the button "up" ▲ or "down" ▼. Press "SET" to confirm the values.

The values that you have set here are then displayed again one after another. The scale then switches itself off automatically at this point.

Activity levels

The medium-term and long-term perspective is decisive when selecting the activity level.

Activity level	Physical activity
1	None.
2	Low: little and light physical exertion (e.g. going for walks, light gardening, simple gymnastics).
3	Medium: physical exertion, at least 2 to 4 times a week for 30 minutes each.
4	High: physical exertion, at least 4 to 6 times a week for 30 minutes each.
5	Very high: intensive physical exertion, intensive training or hard physical work for at least 1 hour every day.

7. Usage

7.1 Taking measurements

Place the scale on an even, hard surface; a hard surface is crucial for achieving accurate measurements.

Measuring your weight, carrying out diagnostics

After all parameters have been entered, weight, body fat and additional values can now be determined.

- Press the “SET” button and select the memory space where you would like to save your basic personal data by pressing the “up” ▲ or “down” ▼ button the appropriate number of times. This is displayed in sequence until “□.□” and the selected gender appear.
- Step onto the scale with bare feet and make sure that you are standing on both electrodes. Your body weight will first be determined and shown.
- Remain on the scale and avoid moving; it is now performing your body fat and body water analysis. This may take a few seconds.

Important: There must be no contact between your feet, legs, calves or thighs. Otherwise, the measurement cannot be taken correctly.

The following data is automatically shown:

1. Weight,
2. Body fat percentage (FAT),
3. Water content (BW),
4. Muscle percentage (MUS),
5. Body mass index (BMI)

All measured values are now shown again successively; the scale then switches off.

Only measuring weight

Step on the scale. It immediately begins to measure. Stand still on the scale with your weight distributed equally on both feet. If the display appears continuously, the measurement has finished.

Switching off the scale

The scale switches itself off automatically.

7.2 Evaluating the results

Body fat percentage

The following body fat values serve as a guideline (for further information please consult your doctor)!

Male

Age	Low	Normal	Moderate	High
10–14	<11%	11–16%	16.1–21%	>21.1%
15–19	<12%	12–17%	17.1–22%	>22.1%
20–29	<13%	13–18%	18.1–23%	>23.1%
30–39	<14%	14–19%	19.1–24%	>24.1%
40–49	<15%	15–20%	20.1–25%	>25.1%
50–59	<16%	16–21%	21.1–26%	>26.1%
60–69	<17%	17–22%	22.1–27%	>27.1%
70–100	<18%	18–23%	23.1–28%	>28.1%

Female

Age	Low	Normal	Moderate	High
10–14	<16%	16–21%	21.1–26%	>26.1%
15–19	<17%	17–22%	22.1–27%	>27.1%
20–29	<18%	18–23%	23.1–28%	>28.1%
30–39	<19%	19–24%	24.1–29%	>29.1%
40–49	<20%	20–25%	25.1–30%	>30.1%
50–59	<21%	21–26%	26.1–31%	>31.1%
60–69	<22%	22–27%	27.1–32%	>32.1%
70–100	<23%	23–28%	28.1–33%	>33.1%

Values will often be lower for fitness enthusiasts. Depending on the sporting activities performed, the training intensity and the physical constitution, the results may still be below the stated standard values. Please note, however, that there may be a risk to health in the case of extremely low values.

Body water content

The body water content in % is normally within the following ranges:

Male

Age	Poor	Good	Very good
10–100	<50%	50–65%	>65%

Female

Age	Poor	Good	Very good
10–100	<45%	45–60%	>60%

Body fat contains relatively little water, therefore body water content may be below the standard values in persons with a high body fat percentage. On the other hand, endurance athletes may exceed the standard values due to having low levels of fat and a high muscle percentage.

The body water calculation performed using this scale is not suitable for drawing medical conclusions, for example in the case of age-related water retention. Consult your doctor where necessary. A high body water content is generally desirable.

Muscle percentage

The muscle percentage is normally within the following ranges:

Male

Age	Low	Normal	High
10–14	<44%	44–57%	>57%
15–19	<43%	43–56%	>56%
20–29	<42%	42–54%	>54%
30–39	<41%	41–52%	>52%
40–49	<40%	40–50%	>50%
50–59	<39%	39–48%	>48%
60–69	<38%	38–47%	>47%
70–100	<37%	37–46%	>46%

Female

Age	Low	Normal	High
10–14	<36%	36–43%	>43%
15–19	<35%	35–41%	>41%
20–29	<34%	34–39%	>39%
30–39	<33%	33–38%	>38%
40–49	<31%	31–36%	>36%
50–59	<29%	29–34%	>34%
60–69	<28%	28–33%	>33%
70–100	<27%	27–32%	>32%

Body weight/body mass index

The body mass index (BMI) is a figure that is commonly used to evaluate body weight. The figure is calculated from body weight and height. The formula is: $\text{body mass index} = \text{body weight} : \text{height}^2$. The measurement unit for BMI is $[\text{kg}/\text{m}^2]$.

Male <20 years

Age	Under-weight	Normal weight	Over-weight
10	12.7–14.8	14.9–18.4	18.5–26.1
11	13.0–15.2	15.3–19.1	19.2–28.0
12	13.3–15.7	15.8–19.8	19.9–30.0
13	13.7–16.3	16.4–20.7	20.8–31.7
14	14.2–16.9	17.0–21.7	21.8–33.1
15	14.6–17.5	17.6–22.6	22.7–34.1
16	15.0–18.1	18.2–23.4	23.5–34.8
17	15.3–18.7	18.8–24.2	24.3–35.2
18	15.6–19.1	19.2–24.8	24.9–35.4
19	15.8–19.5	19.6–25.3	25.4–35.5

Female <20 years

Age	Under-weight	Normal weight	Over-weight
10	12.7–14.7	14.8–18.9	19.0–28.4
11	13.0–15.2	15.3–19.8	19.9–30.2
12	13.3–15.9	16.0–20.7	20.8–31.9
13	13.7–16.5	16.6–21.7	21.8–33.4
14	14.2–17.1	17.2–22.6	22.7–34.7
15	14.6–17.7	17.8–23.4	23.5–35.5
16	15.0–18.1	18.2–24.0	24.1–36.1
17	15.3–18.3	18.4–24.4	24.5–36.3
18	15.6–18.5	18.6–24.7	24.8–36.3
19	15.8–18.6	18.7–24.9	25.0–36.2

Male/female ≥20 years

Age	Under-weight	Normal weight	Over-weight
≥20	15.9–18.4	18.5–24.9	25.0–40.0

Source: Bulletin of the World Health Organization 2007;85:660-7; Adapted from WHO 1995, WHO 2000 and WHO 2004.

Please be aware that your BMI results may show you to be overweight if you have a very muscular body (body-builders). The reason for this is that the significantly above average muscle mass is not taken into account in the BMI formula.

Temporal context of the results

i **Note:** Please note that only the long-term trend is important. Brief deviations in weight within a few days are normally caused by loss of fluids.

The interpretation of the results is based on changes in total body weight, percentage of body fat, body water and muscle content, as well as on the length of time over which these changes occur.

Rapid changes within the scope of a few days are to be considered separately from medium-term changes (in the scope of weeks) and long-term changes (months).

It can be said as a basic rule that short-term changes in weight are almost entirely changes in water content, whereas medium-term and long-term changes may also relate to the fat percentage and the muscle percentage.

- If your weight drops in the short term but your body fat percentage increases or stays the same, you have only lost water, for example following a training session, visit to the sauna or a diet aimed only at fast weight loss.
- If your weight increases in the medium term and your body fat percentage drops or stays the same, you may have built up valuable muscle mass.
- If your weight and body fat percentage both fall at the same time, then your diet is working—you are losing fat mass.
- Ideally, you should support your diet with physical activity, fitness or strength training. This enables you to increase your muscle percentage in the medium term.
- Body fat, body water and muscle percentages should not be totalled (muscle tissue also contains components made of body water).

8. Cleaning and maintenance

The device should be cleaned from time to time.

This should be done using a damp cloth and, if necessary, a small amount of detergent.

IMPORTANT

- Never use abrasive solvents or cleaning products!
- Never submerge the device in water!
- Do not clean the device in a dishwasher!

9. What if there are problems?

Display	Cause	Solution
Err 2	The proportion of fat is outside the measurable range (less than 5% or greater than 80%).	Please repeat the measurement barefoot, or with the soles of the feet lightly dampened if necessary.
Err	Maximum weight capacity of 180 kg exceeded.	Loads must not exceed 180 kg.
Err or incorrect weight is displayed.	No even, hard surface.	Place the scale on an even, hard surface.
Err or incorrect weight is displayed.	Not standing still.	Stand as still as possible.
Incorrect weight is displayed.	Incorrect zero point of scale.	Wait until the scale switches itself off. Activate the scale, wait for "0.0 kg" to appear and take the measurement again.
L0	The batteries in the scale are flat.	Replace the batteries in the scale.

10. Disposal

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = battery contains lead,

Cd = battery contains cadmium,

Hg = battery contains mercury.

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

11. Technical specifications

Model:	SR BF 2
Dimensions:	30 x 30 x 1.8 cm
Weight:	Approx. 1.6 kg
Measurement range:	5 – 180 kg
Scale interval d:	The display can be read in increments of 0.1 kg.
Repetition accuracy:	The measuring tolerance for repeated measurements is ± 0.4 kg (several measurements in direct succession on the same scale with position of scale and person the same as far as possible).
Absolute precision:	In comparison with a calibrated weight, the measured value is $\pm 1\% + 0.1$ kg. E.g. at 40 kg this corresponds to ± 0.5 kg; at 100 kg this corresponds to ± 1.1 kg.

Subject to technical changes.

12. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as fo

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Beurer GmbH, Söflinger Strasse 218, D-89077 Ulm (di seguito denominata “Beurer”) offre una garanzia per questo prodotto
Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany • www.beurer.com
Beurer UK Ltd., Suite 16, Stonecross Place, Yew Tree Way,