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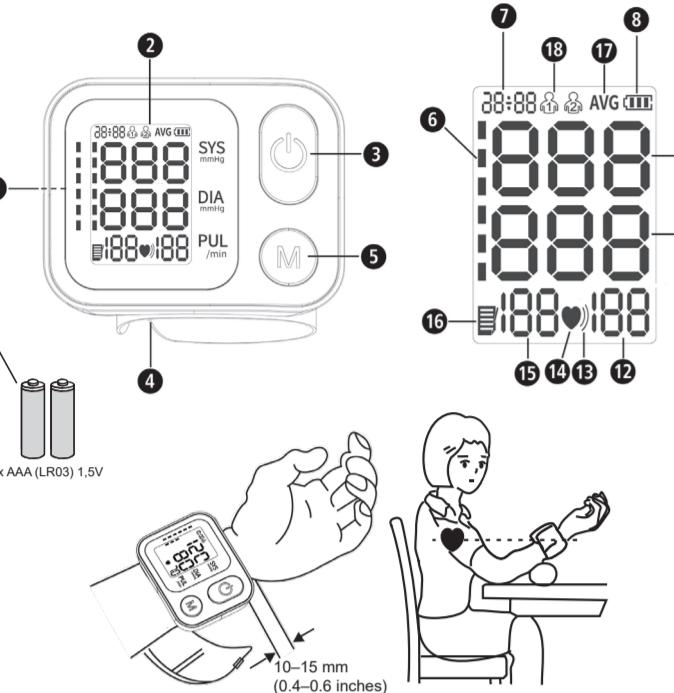
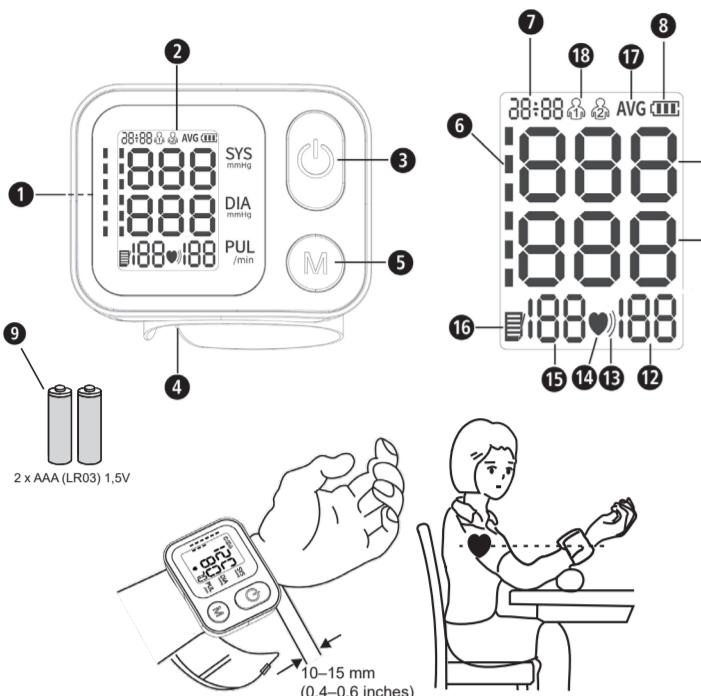
#### Blutdruckklassifikation</h4

GB Instruction Manual  
Wrist blood pressure monitor BW 345

#### Device and controls

#### Explanation of Symbols

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

These warnings must be followed to prevent possible injury to the user.

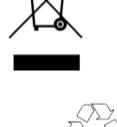
**CAUTION**  
These instructions must be followed to prevent possible injury to the device.

**NOTE**  
These instructions provide you with useful additional information regarding installation or operation.

The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects.

This product is subject to the European Directive 2012/19/EU on waste electrical and electronic equipment and is labelled accordingly. Disposal only according to the "WEEE" Waste Electrical and Electronic Equipment EC Directive.

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

Device complies with European Regulation on Medical Device EU MDR 2017/745.

REF	Article Number		General prohibition symbol
SN	Serial Number		Do not disassemble or dismantle
LOT	LOT number		This side up
MD	Medical Device		Device classification: type BF applied part
EC REP	EU authorized representative		Keep dry
UDI	Unique Device Identifier		Humidity range
	Keep away from sunlight		Ambient pressure limitation
	Manufacturer		Temperature range
	Date of manufacture		Importer

Name: BW 345 [Type WRS-35H]

REF 51073

EAN 4015588 51073 1

Manual

Service

## GB IMPORTANT INFORMATION! KEEP IN A SAFE PLACE!

Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference. If you pass the device on to a third party, this instruction manual must remain with the device.

Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.

The unit is intended only for use in the home.  
Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.  
The unit may not be used to check the heart rate of a pacemaker.

The result of a blood pressure measurement with this device is not suitable for self-diagnosis or self-treatment. To avoid health risks, always follow your doctor's instructions and contact him/her for an assessment of the measurement result.

Never change the intake of medication prescribed by a doctor as a result of a measurement with this device and consult your doctor for advice on the best time to take the measurement before measuring.

The device is not suitable for continuous monitoring of a patient, assessment of an existing disease or first aid.

The device is not suitable for use on patients undergoing surgical treatment and should only be used in specified locations or in private homes.

Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.

The patient is an intended operator and all the functions can be safely used.

If you experience discomfort during a measurement, such as pain in the wrist or other complaints, press the Start/Stop button ① to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.

This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.

This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons. Medical devices are not toys!

Do not let a child below 12 years old or the people who can't express intention to use the device.

Keep the appliance out of the reach of children.

The device must not be operated in rooms with high radiation levels or in the vicinity of devices with high radiation levels, such as radio transmitters, mobile phones or microwaves, or together with high-frequency surgical devices (magnetic resonance [MR] devices). This may cause malfunctions or incorrect readings.

Do not operate near active RF surgical equipment and the RF-shielded room of an ME system for magnetic resonance imaging where the intensity of EM interference is high.

Do not move during the measurement.

Protect the unit from moisture. Should liquid nevertheless penetrate the device, the batteries must be removed immediately and further applications avoided. In this case, contact your specialist dealer or inform us directly.

Never use the unit outdoors or in damp rooms.

Do not operate the unit near flammable gas (e.g. narcotic gas, oxygen or hydrogen) or flammable liquid (e.g. alcohol).

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Do not make any modifications to the unit, do not disassemble it and never repair the unit yourself. Have repairs carried out only by authorised service centres.

Before using the device, it is the user's responsibility to determine that the device is safe and working properly.

Repeated blood pressure measurements in high numbers can lead to undesirable side effects, e.g. nerve crushing or blood clots.

Do not measure more than 3 times in a row and leave at least 5 minutes between two measurements. Do not measure more than 6 times a day in total.

Never place the cuff over injured skin, with an existing catheter connection or after a mastectomy. Always keep an eye on the pressure reading on the display. If it exceeds 295 mmHg (39.3 kPa), press the Start/Stop button ① immediately to deflate the cuff.

The cuff is delicate and must be handled with care. Only inflate the cuff when it is correctly applied to the wrist.

Protect the unit from heavy blows and do not drop it. Never use force, e.g. when putting on the cuff.

Use only original accessories and spare parts from the manufacturer, otherwise damage to the unit or personal injury may result.

Observe the local disposal guidelines if you wish to dispose of the unit.

If you have any doubts about the proper functioning of the unit, stop using it and contact the service centre immediately.

Swallowing small parts such as packaging material, battery, battery compartment cover, etc. can lead to suffocation. Therefore, never leave children alone with the unit.

No maintenance or cleaning may be carried out during operation. Follow the maintenance and cleaning instructions in this manual.

People with irregular or unstable peripheral blood circulation due to diabetes, liver disease, arteriosclerosis, etc. may experience fluctuations in the measured blood pressure values.

Wash your hands before using the device.

Do not measure blood pressure when other measurements are being taken on the same part of the body at the same time, as these may be disturbed or fail.

If the ambient temperature is less than 5°C or more than 40°C, the waiting time from minimum storage temperature to use is at least 1 hour; the time from maximum storage temperature is at least 2 hours.

Blood pressure values measured with this device are usually slightly lower than clinically determined values due to the quiet conditions.

Remove the batteries if you are not using the device for a long time.

Too frequent measurements can cause injury to the patient due to blood flow interference.

The cuff has been tested for biocompatibility, but it varies from person to person, and some people may still be allergic to the contact material.

Maintenance should be done by the operator as suggested.

When the performance changes (such as: inaccurate measurement or abnormal display), please stop using it immediately and contact the after-sales service personnel in time.

#### SAFETY NOTES FOR BATTERIES

Do not disassemble batteries!

Replace all batteries if the unit display shows the low battery symbol.

Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.

Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!

If a battery has been swallowed, seek medical attention immediately!

Replace all of the batteries simultaneously!

Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!

Insert the batteries correctly, observing the polarity!

Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!

Keep batteries out of children's reach!

- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

#### Unit and LC-Display

① Battery compartment	② LC-Display	③ START/STOP button	④ Wrist cuff
⑤ User selection / Memory / Settings	⑥ Low battery	⑦ Blood pressure indicator	
⑧ Time / Date	⑨ Diastolic blood pressure	⑩ Pulse rate	
⑩ Systolic blood pressure	⑪ Pulse icon	⑫ Memory number	
⑫ Irregular heartbeat icon	⑬ Average value	⑭ User icon 1 / 2	
⑬ Memory icon			

#### Scope of delivery

The scope of delivery includes:

- 1 medisana Wrist blood pressure monitor BW 345
- 2 Batteries (type AAA, LR03) 1.5V
- 1 Storage box
- 1 Instruction manual

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your supplier or your service centre.

If you notice any transport damage during unpacking, please contact your supplier without delay.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

#### Intended use

The Wrist Blood Pressure Monitor is intended to measure the systolic pressure and diastolic pressure, as well as the pulse rate of adult person via non-invasive oscillometric technique by an inflatable cuff wrapped around the wrist. The unit can be used in medical facilities or at home.

**Contraindications:** There are no known contraindications.

#### What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

#### Blood pressure classification

systolic mmHg	diastolic mmHg	Classification
≥ 160	≥ 100	High blood pressure
140 - 159	90 - 99	Mild hypertension
90 - 139	60 - 89	Normal blood pressure
< 90	< 60	too low blood pressure

**WARNING!**  
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

#### Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

#### Starting up: Insert / replace batteries

Before you can use your device, you must insert the enclosed batteries. The cover of the battery compartment is located on the back of

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