VULPES GOODS
BABYCARE

ELECTRIC NASAL ASPIRATOR

**MANUAL • ANLEITUNG
HANDLEIDING • MANUEL
MANUAL • MANUALE • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH	4
The Device	5
User Guide	8
Characteristics and Application	9
Product parameters	12
DEUTSCH	13
Das Gerät	14
Benutzerhandbuc	17
Eigenschaften und Anwendung	18
Produkt parameter	21
NEDERLANDS	22
Het apparaat	23
Gebruiksaanwijzing	26
Kenmerken en Toepassingen	27
Product Parameters	30
FRANCAIS	31
Le dispositif	32
Manuel de l'utilisateur	35
Caractéristiques et applications	36
Paramètres du produit	39

ESPAÑOL **40**

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Music button

Suction head

You can use the 3 different attachments for different blockages. For example, deep mucus, shallow mucus and hard pieces.

Collection cup

The collection tray collects all residues. Clean the collection tray on time. The nasal cleaner will not work without a collection tray.

Led screen

Use the LED screen to read the mode and battery status.

On/Off button

Press once to turn the electric nasal aspirator on, then press again to turn it off.

Gear button

Pressing the button selects one of the five available suction levels. Each click increases the suction power. After suction level 5, the suction level goes back to 1.

Light button

Press the light button once to turn on the light and illuminate the nasal cavity. Press again to turn it off.

Music button

If you press the music button, 3 different songs can be played. Each time you press the button, the song changes. After the third song, it will go back to mute.

USER GUIDE

Before using the nasal cleaner for the first time, it is important to make sure the cleaner is fully charged for optimal performance.

1. Place the collection cup on the nasal cleaner and choose the appropriate attachment
2. Press the power button to activate the nasal cleaner and choose the desired suction strength.
3. For an optimal experience, you can use the music button and choose from 3 different children's songs.
4. For an even better experience, you can also use the light button. This is also possible while using the music button.
5. After using the nasal cleaner, clean the used attachments and collection cup with lukewarm water.

CHARACTERISTICS AND APPLICATION

- Do not expose this product to high temperature, high humidity or direct sunlight for a long time.
- Prevent the occurrence of intense pressure collisions or accidental drops of this product.
- This product is only suitable for absorbing nasal secretions. Do not use it for improper use, such as absorbing water or skin.
- When the suction is significantly reduced or the motor sound is significantly reduced, please charge it in time.
- When using this product, make sure the user does not hold their breath or swallow saliva. Keep the mouth open to avoid pressure differences in the ear.
- In Case of any pain or discomfort during use. Stop immediately, and seek medical attention!
- Avoid inserting the suction head into the nasal cavity with excessive pressure.

- Clean the suction head before each use and keep it clean.
- Adults must adhere to the operating instructions when using it, and it is prohibited for children to operate it independently. Ensure the product is stored in a location inaccessible to children.
- Not possible to put your own music on the product.
- Do not put product in dishwasher and do not submerge in water or liquid.

- Some weak slime can flow back (If you are able to clear the suction port of the main engine using a cotton swab).

- Utilize this product in the correct posture/ a method to prevent liquid from being drawn into the main device.

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