VULPES GOODS
HEALTHCARE

MESH NEBULIZER

**MANUAL • ANLEITUNG
HANDLEIDING • MANUEL
MANUAL • MANUALE • MANUAL**

WWW.VULPESGOODS.COM

ENGLISH **4**

The Device	5
User Guide	8
Characteristics and Application	10
Product parameters	13

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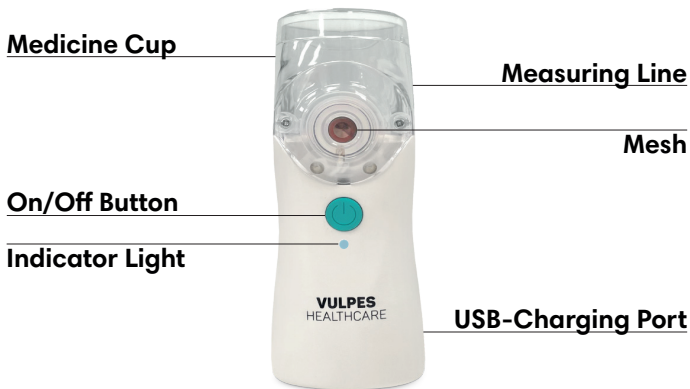
THE DEVICE

Medicine Cup

Fill the medicine cup with the medicine or liquid and ensure it is properly connected. If the medicine cup is not properly connected, the nebulizer will not work.

Measuring Line

Helps measure the amount of liquid. Ensure the liquid does not exceed the maximum amount of 20 ml.

Mesh

Keep the mesh free of residue and ensure proper cleaning after use. This enhances the device's lifespan.

On/Off Button

Hold the on/off button for 3 seconds to turn on the nebulizer. Press once to set the nebulizer to stand 2, which will cause it to nebulize more slowly. Press again to turn off the nebulizer.

USB Charging Port

Use the included USB charging cable to charge the nebulizer.

Adult Mask

Suitable for adults. Less medication is inhaled per breath through the mask than when using the mouthpiece.

Child Mask

For babies and children, the mask must cover both the mouth and nose for effective inhalation.

Mouthpiece

Use the mouthpiece to inhale the medication directly for efficient inhalation.

Indicator Light

Usage

Blue:	On
Blue-Green Pulsing:	Stand 2 is activated
Blue Pulsing:	Battery is nearly empty

Charging

Blue Pulsing:	Charging
Green:	Fully charged

USER GUIDE

Ensure the nebulizer is fully charged before first use.

- 1.** Unscrew the lid of the medicine cup and fill it with the chosen medicine or liquid. Do not exceed the maximum amount. After adding the medication or liquid, screw the lid back on tightly.
- 2.** Attach the chosen mouthpiece or mask to the nebulizer.
- 3.** Turn on the nebulizer by holding the on/off button for 3 seconds. Press the on/off button again to set the nebulizer to stand 2. The nebulizer will then nebulize more slowly. Place the chosen mouthpiece or mask in the correct position after the first nebulization.
- 4.** Stay calm, relax, and sit upright. Hold the nebulizer comfortably, breathe in deeply and slowly. This ensures a more effective and pleasant treatment.

Cleaning

- 1.** Open the medicine cup and dispose of any remaining medication or liquid. Pour a small amount of clean water (10-60°C) into the medicine cup and close the lid.
- 2.** Turn on the nebulizer and allow the water to nebulize for 3 to 5 minutes to remove any remaining medication from the mesh.
- 3.** Rinse the medicine cup and the used mask or mouthpiece with clean water. Allow the parts to air dry in a clean environment.
- 4.** For extra cleaning, the inside of the medicine cup can be disinfected with 75% medical alcohol. Afterward, clean the device again as described above.
- 5.** Store the nebulizer in the provided storage bag or in a clean environment, out of reach of children, away from direct sunlight, and not in damp areas.

CHARACTERISTICS AND APPLICATION

- To prevent cross-contamination, the device should be used by one patient only
- Before cleaning the device, always remove the charging cable.
- Do not store the device in a damp area (e.g., bathroom) and do not transport it with wet items.
- Store the device away from direct sunlight.
- Proper breathing technique is important to ensure the medication particles are widely dispersed through the airways. To ensure this happens, breathe in deeply and slowly for about 5 to 10 seconds and then exhale quickly.
- Do not dry the device or parts with a microwave, oven dryer, hairdryer, etc.
- Use medication as prescribed.
- Keep the device out of reach of children.
- Do not submerge the device in water or any other liquid.
- Use the device only for its intended purpose and as described in the manual.
- Do not use the device if you have an active skin condition.

- Do not use the device while it is charging. The nebulizer may not function optimally while charging.
- Irregular breathing can cause a blockage in the airways.
- Do not swallow the medication or liquid, and ensure it does not directly reach the affected area.
- Only use the prescribed medication or water. Do not use oils or thickened medications.
- For medical advice, we always recommend consulting a doctor or qualified expert.
- Clean the device properly to avoid damage.


The nebulizer does not turn on.	1. The battery is empty. Charge the nebulizer. 2. The nebulizer is damaged.
The nebulizer produces little or no mist.	1. The nebulizer is not being held correctly. Ensure the nebulizer is held upright. 2. The liquid is too little or unsuitable. Refill or replace the liquid. 3. There are too many air bubbles in the liquid. Gently shake the nebulizer. 4. The medicine cup has not been cleaned properly. Clean the medicine cup. 5. The device is damaged.
Which medications are suitable?	Only a doctor or medical expert can advise which medications are suitable for you.
Why does some medication/liquid remain in the medicine cup?	This is normal. Clean the medicine cup after use to prevent blockages.
The medicine cup leaks.	The medicine cup is broken or the plastic is outdated. Replace the medicine cup.
Water comes into the mouthpiece.	Due to the temperature difference, the mist turns into water droplets on the mouthpiece. Remove the mouthpiece and pour out the water.

PRODUCT PARAMETERS

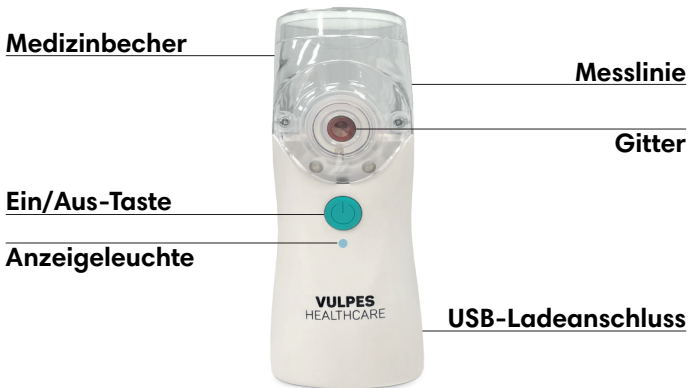Weight:	100 Gr
Dimensions:	42 x 55 x 119 mm
Power Consumption:	1.5 W
Vibration Frequency:	110 KHz $\pm$ 10 KHz
Nebulization Speed:	≥ 0.3 ml/min
Volume:	max. 20 ml
Operating Temperature:	5 ~ 40 °C
Storage Temperature:	-25 ~ +70 °C
Relative Humidity:	<90%

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