InnovaGoods®

User's Manual

ENGLISH

CONTENTS/TECHNICAL SPECIFICATIONS

- Hand Rail for Bed InnovaGoods
- 2 bolts
- 4 screws
- 4 nuts
- 2 adjustable straps

INSTRUCTIONS

- Follow the assembly guide for the Hand Rail for Bed InnovaGoods as shown in the illustrations.
- Attach the horizontal base to the vertical base by matching the holes and fastening it with the supplied screws and nuts (1).
- Insert the support handle into the vertical base (2). Adjust to the desired height and assemble the latches (3), ensuring they align with the holes.
- Remove the mattress, and place the horizontal base of Hand Rail for Bed InnovaGoods on the mattress bed or upholstered base in the area where it is required as a support. Secure it firmly with the 2 straps.
- It is advisable to loop the straps over the base plate for greater support and to avoid slippage. Replace the mattress.

WARNINGS

Make sure that the Hand Rail for Bed InnovaGoods is securely attached to the mattress bed or upholstered base. Always check before each use. Wipe with a slightly damp cloth and a non-abrasive soapy solution. Suitable for domestic use only. Check correct assembly before use. Read the instructions carefully before first use. Keep the manual for further queries.

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