
Relaxing Toe Separator x2 InnovaGoods -

after standing for long periods, wearing tight-fitting footwear, excessive training or impact sports activity, etc. They also help to regain mobility by improving the foot strike and preventing foot-related problems.

- You can cool the Relaxing Toe Separator x2 InnovaGoods in the refrigerator or heat them with warm water before us to obtain a more pleasant cooling/heat effect.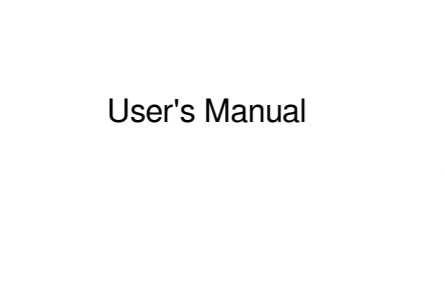

User's Manual

ENGLISH

WARNINGS

IMPORTANT: KEEP THE INSTRUCTIONS FOR

FUTURE REFERENCE. This device is not designed for use by children or people with physical and mental disabilities, unless they are fully supervised by an adult who is responsible for their safety. Read the instructions carefully before first use. Keep the manual for further queries.

SPECIFICATIONS

- Relaxing Toe Separator x2 InnovaGoods: 2 pairs with different degrees of hardness
- One size: static and flexible, they adjust easily
- Reusable and washable
- Material: silicone

INSTRUCTIONS

- Choose between the 2 pairs included of the Relaxing Toe Separator x2 InnovaGoods: the high profile pair with increased separation (particularly the big toe), or the flat and extra-soft profile pair, with a natural separation and equal distribution of the toes. Place them between the toes until they fit completely in the space between the toes.
- The Relaxing Toe Separator x2 InnovaGoods adapts to the physiological foot structure to prevent toes from overlapping and correct posture.
- Actively use them as part of your daily routine, doing moderate physical activity to improve their effectiveness, such as walking, functional or weight exercises, etc. Recommended for use barefoot, for example, while at home. They can also be used with wide-fitting footwear, sandals, socks, etc.
- To start with, use the Relaxing Toe Separator x2 InnovaGoods for approx. 5-10 minutes and gradually increase by a few minutes with each use, until you can wear them comfortably for a longer period.
- If they are only used during periods of rest, their effects will be slower and more limited. Equally, regular use of suitable footwear is advised to ensure their benefits are not counteracted.
- The Relaxing Toe Separator x2 InnovaGoods alleviate pressure and pain

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