InnovaGoods®

User's Manual





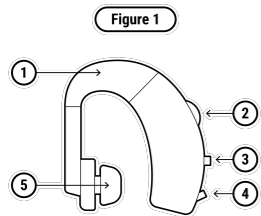
Figure 2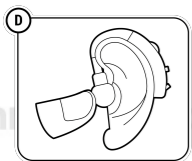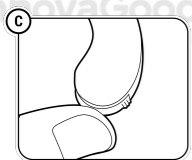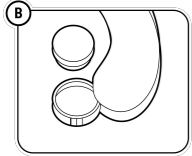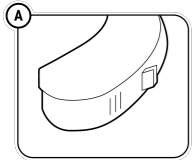

ENGLISH

WARNINGS

IMPORTANT: KEEP THE INSTRUCTIONS FOR

FUTURE REFERENCE. The amplifier should only be used to improve a person's hearing ability. The device has been designed solely for the use described in these instructions. InnovaGoods® shall not be liable for any damage resulting from its improper or

irresponsible use. Before using the sound amplifier, see a doctor or hearing specialist so they can check your hearing. If you are suffering from any ear condition or have suffered from a condition in the past, check with your doctor before using the hearing aid. It is NOT permitted for use in patients suffering from ear inflammation or infection. DO NOT use the amplifier if you are suffering from a middle ear infection (otitis media), stop using it and see your doctor. If you feel any pain or discomfort when wearing the amplifier, stop using it and see your doctor. If the ear becomes inflamed, the skin becomes irritated or you have an increased amount of earwax when wearing the amplifier, stop using it and see your doctor. Do not use for children. Do not wet or immerse the device in water or use near electromagnetic sources. If any part of the device gets stuck inside the ear, see a doctor, do not try to remove it yourself. Keep the device away from children, as it contains small parts. Remove the battery if you are not going to use the device. Do not use the device on a high volume for too long, it could damage your hearing. Do not use in damp or high temperature environments (like a sauna). If you are allergic to silicone, you should not use this device. Do not use a mobile telephone near this device, as it may cause interference. Retain the manual for future reference. Do not mix new and used batteries in the compartment. If you are not going to use the unit for a long period of time, remove the battery. Do not expose batteries to high temperature or direct sunlight. Do not attempt to open them, do not dispose of them in fire, there is risk of explosion. Clean with a damp cloth. Do not use abrasive products for cleaning. Do not immerse the appliance in liquid. Do not handle it with wet hands Do not use near water or damp areas. Suitable for domestic use only. Use the device for its intended purpose only. Store the device in a dry, dust-free place. Do not leave the device unsupervised while in use. To avoid risks, do not use it if any parts are damaged and do not attempt to disassemble or repair it. This device is not intended for use by children or persons with reduced physical, sensory or mental capabilities, unless they are supervised by an adult responsible for their safety. This device is not household waste. Please follow the environmental regulations in place for its recycling Read the instructions carefully before first use. Keep the manual for further queries. This product contains a coin cell battery. Ingestion of a coin cell battery can cause internal chemical burns, soft tissue perforation, and death. Severe burns may occur within two hours of ingestion. Dispose of used batteries immediately. Keep new and used batteries out of reach of children. If you think the batteries have been swallowed or introduced to any part of the body, seek immediate medical attention. Do not allow children to change batteries without adult supervision.

TECHNICAL SPECIFICATIONS

- Battery: 1 x A13
- Max. OSPL 90: ≤128dB+3dB
- Max. sound gain: ≤37dB+3dB
- Total harmonic distortion: ≤7%+3%
- Frequency range: 320-3200 Hz
- EQ input noise: ≤33dB+3dB
- Operating current: ≤4.5 mA
- Battery life: up to 85 hrs
- Volume control
- «ON/OFF» button
- Use in both ears
- 2 cleaning accessories, 4 silicone ear pads, 1 replacement battery

INSTRUCTIONS

- This product is designed to improve people's hearing ability, but it does not prevent or mitigate hearing loss related to organic causes. However, it helps you to hear the surroundings with greater clarity and sharpness.
- For the first few uses, it is normal to feel some discomfort in the ear until you become accustomed to using it.
- Open the compartment (4-A) by pushing the plastic tab with your nail. Remove the plastic from the battery and wait for 1 min before installing it. Insert the battery as shown in figure (B) with the correct polarity. Close the compartment (4-C).
- Place the silicon ear pad (5) which best fits your ear.
- Switch the device (3) on and adjust the volume to minimum (2). Position the Hearing Amplifier InnovaGoods® in the ear (D). The flexible arm (1) enables you to use it in both ears.
- Gradually increase the volume (2) between 1 (min.) and 4 (max.), until you can hear the surroundings more clearly. If you hear beeping/feedback, lower the volume on the device (this happens when the outer volume and the volume of the Hearing Amplifier InnovaGoods® are not aligned correctly or there is a source of interference nearby). Try to find a balance between an improvement in your hearing and the lowest volume on the device; this will prevent feedback and will extend the battery life.
- When you are not using the device, remember to switch it off using button (C).
- In most cases, users will require a period of adjustment to the new hearing amplifier system.
- Clean the silicone plug with the cotton bud included and with a slightly damp cloth.
- Change the battery when the Hearing Amplifier InnovaGoods® loses amplification power.

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