## ENGLISH

### WARNINGS

**IMPORTANT: KEEP THE INSTRUCTIONS FOR FUTURE REFERENCE.** This appliance is not a toy. Keep out of reach of children. Fully charge the battery before its first use. Make sure that the voltage of the power source corresponds to that of the appliance. Do not pull on the cable and ensure it does not tangle. Keep the cable away from hot and sharp surfaces. Clean with a damp cloth. Do not use abrasive products for cleaning. Do not immerse the appliance or cable in liquid. Do not handle it with wet hands. Do not use near water or damp areas. Suitable for domestic use only. Use the device for its intended purpose only. Store the device in a dry, dust-free place. Do not leave the appliance unsupervised when it is turned on. If the appliance, the cable or the pieces looked damaged or do not work correctly, do not use them or attempt to repair them; they should only be repaired by a qualified technician. This device is not intended for use by children or persons with reduced physical, sensory or mental capabilities, unless they are supervised by an adult responsible for their safety. This device is not household waste. Please follow the environmental regulations in place for its recycling. Discard the product in accordance with the European Directive 2002/96/EC. Enquire about your local recycling legislation for electrical and electronic products and follow these steps. Do not dispose of this product with household waste. Following these instructions will help prevent harmful consequences to personal and environmental safety.

### TECHNICAL SPECIFICATIONS

- DC In: 5 V / 1 A
- Battery: Li-ion 500 mAh
- 6 speeds
- 2 silicone tips
- 10 reusable filters
- USB cable

### INSTRUCTIONS

- Connect (1) the Rechargeable Baby Nasal Aspirator InnovaGoods® to a USB source to charge it.
- Before using it for the first time, disinfect the heads (4) with a hot saline solution for 10 minutes.
- Remove the head (2), insert 1 sponge filter into the compartment (3) and close it again.
- Place the most suitable silicone tip (4). The curved tip is for children aged 0 to 2 years. The triangular tip is for children from the age of 2 years.
- Apply a saline spray or solution (consult your doctor or pharmacist beforehand) to the nasal cavity and leave to activate for 5 minutes to soften the mucus.
- The child should be in an upright position, avoid laying them down horizontally. Switch on the Rechargeable Baby Nasal Aspirator InnovaGoods® using the button (5) and select the lowest suction speed (6). Insert the silicone tip into the child's nostril, leaving the device positioned at a 45° angle (see illustration). Suck for 2 seconds, remove for 2 seconds and insert it again for another 2 seconds. If that isn't enough, increase the suction speed by pressing the button (5). Don't force the silicone tip into the child's nose. If necessary, return to point 6 and repeat the

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