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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the VITALPRESS digital wrist blood pressure monitor! In this manual you will find all the information you need to use and maintain this product correctly and safely.

This blood pressure monitor has among its main functions automatic measurement, an easy-to-read digital display, arrhythmia detection and memory for storing 99 measurements.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

- 1. WARNING!** Do not use the cuff on an arm that is injured or receiving intravenous fluids.
- 2. WARNING!** Do not use the device if the arm where intravascular access or therapy, or an arteriovenous (A-V) shunt is present due to temporary interference with blood flow as it may cause injury to the patient.
- 3. WARNING!** Do not use the cuff on the arm on the side of a mastectomy or lymph node cleansing.
- 4. WARNING!** Note that when the cuff is inflated, it will simultaneously lose the function of other monitoring devices on the same limb.
- 5. WARNING!** Do not perform frequent measurements as restriction of blood flow may cause injury.

2. PURPOSES OF THE PRODUCT

- **Measuring blood pressure:** Check your blood pressure at any time.
- **Arrhythmia detection:** This model measures the pulse, thus detecting irregularities in the heart rhythm.
- **Portable and easy to use:** Compact, battery-powered, and easy to operate without technical knowledge.
- **Save results:** Stores previous measurements for follow-up.

3. PRODUCT FEATURES


| | |
|------------------------------|--|
| PRODUCT NAME | Digital wrist blood pressure monitor |
| MODEL | VITALPRESS |
| MODEL REFERENCE | E-00072/36 |
| PRODUCT WEIGHT | 0,22 lb |
| PRODUCT DIMENSIONS | <ul style="list-style-type: none"> • With sleeve: 3,50"*2,44"*2,55" • Without sleeve: 3,50"*2,44"*0,90" • Wrist cuff: 5,31"- 7,67" |
| MATERIALS | <ul style="list-style-type: none"> • ABS rear cover • ABS top cover • Velcro: Polyamide • Fabric: Polyamide, polyester and elastane • Panel: PC |
| OTHER CHARACTERISTICS | <ul style="list-style-type: none"> • Small and delicate design • Capacity to store up to 99 measurements • Display of the average value of three measurements • Different units of measurement: kPa and mmHg |

TECHNICAL PARAMETERS:


- Display: LCD digital display.
- Operating principle: Oscillometric method.
- Measurement range:
 - Diastolic: 40 mmHg - 210 mmHg.
 - Systolic: 60 mmHg - 260 mmHg.
 - Cuff pressure: 0 mmHg ~ 300 mmHg.
 - Pulse rate: 40 beats/min ~ 200 beats/min.

- Accuracy:
 - Pressure: within ± 3 mmHg (± 0.4 kPa).
 - Pulse rate: within 5% of reading.
 - Working system: continuous operation.
- IP rating: IP22
 - "2" Protected against solid foreign bodies 0,49" \varnothing and larger.
 - "2" Protection against vertically falling drops of water with the enclosure inclined at an angle of up to 15°.
- Electrical classification: Class II and internal power supply, type BF applied part (sleeve is applied part)
- Power supply:
 - Battery: 2 * 1.5V AAA batteries (not included).
 - Battery life: the battery can be used up to 300 times.

4. PARTS AND COMPONENTS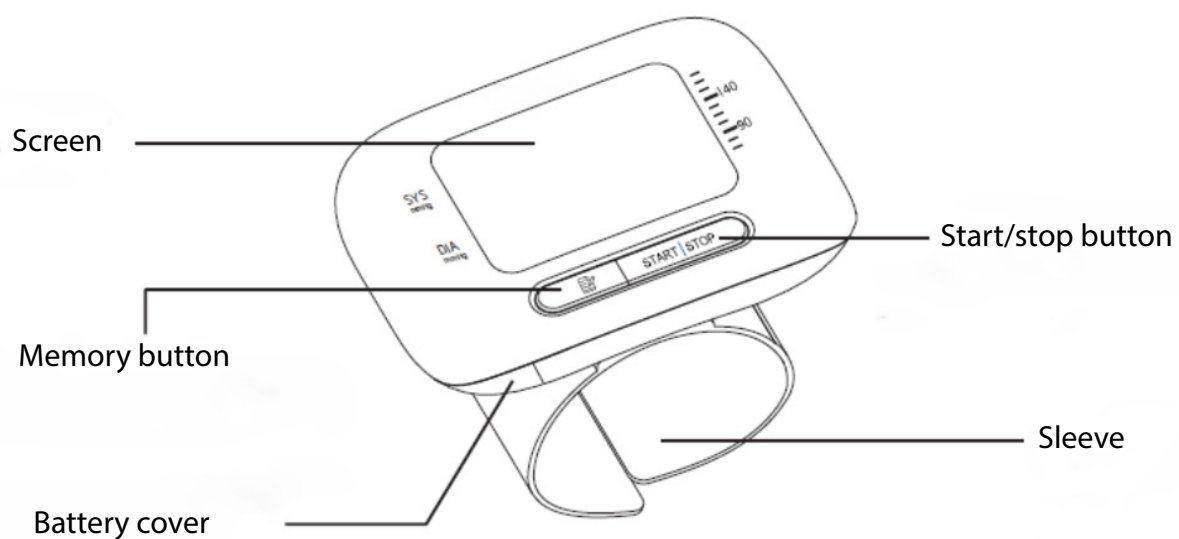

DISPLAY

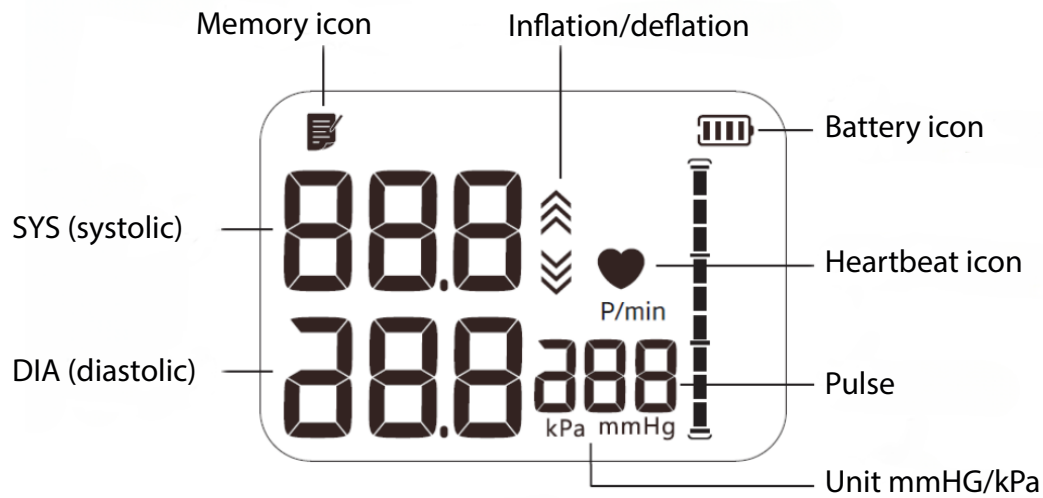
5. INDICATIONS

This product is intended to measure blood pressure and pulse in adults over 12 years of age with a wrist circumference between 5,31" and 7,67" at home or in a medical facility.

6. INSTRUCTIONS FOR USE

Cuff method of use: Do not measure other parts of the body, this blood pressure monitor should be worn on the wrist.

1. Place the cuff on the wrist, do not leave clothing inside the cuff. Keep the cuff neither too loose nor too tight.
2. Suitable wrist circumference is 5,31"~7,67"
3. The distance between the palm of the hand and the end of the monitor should be approximately 0,39"~0,59".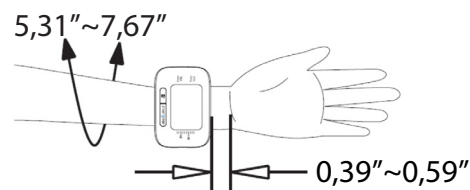


4. Attach the cuff firmly. Or it may loosen during measurement and affect the results.
5. Roll up the sleeve correctly.

Correct measuring posture:

Note: Any blood pressure measurement is influenced by the posture and physical condition of the person being measured.

1. Sit up straight, place your arm on the table with palm up and keep the centre of the cuff at the same height as your heart.
2. Do not bend over or lean forward.
3. Take 3~4 deep breaths and relax your shoulder and arm before measurement.
4. Do not talk, shake your hand or bend your fingers during measurement.
5. It is advisable to support the arm with a soft pillow. Your back should also be supported
6. Legs uncrossed with feet flat on the ground.

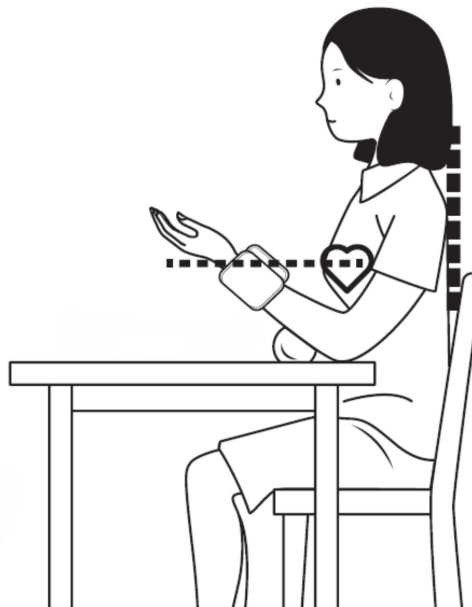**Incorrect measuring posture:**


1. Do not clench the cuffs, or you may cause an increase in blood pressure.
2. Do not hold the cuff with the other hand.
3. Do not hold the arm tightly during the measurement. If necessary, rest the arm on a soft pillow.

Installation of batteries:

1. Open the battery cover in the direction of the arrow.

Note: Press the two sides of the battery cover firmly together and pull it off.

2. Install two AAA batteries correctly.
3. Close the lid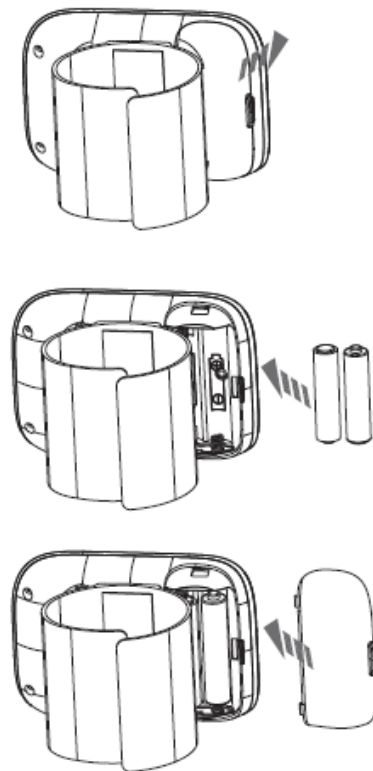

- If battery electrolyte accidentally comes into contact with eyes, skin or cloth, rinse with plenty of water.
- The "🔋" icon means the battery is low. Replace all batteries.
- Remove the batteries if the monitor will not be used for a long period of time (more than three months).
- Do not mix new and used batteries.
- Dispose of batteries in accordance with local environmental legislation.

TIPS FOR USE

- Sit still for 5 minutes before measuring to ensure a calm and stable state.
- Do not measure while standing, walking or with the body under pressure.
- Do not measure after smoking, drinking coffee or tea.
- Do not measure after exercise or bathing.
- Do not talk or move during the measurement.
- Do not move, shake the arm or bend the fingers during measurement.
- Do not measure in extreme temperature conditions or in environments that change drastically.
- Do not measure in a moving vehicle.
- Do not measure continuously (you should leave about 5 minutes between two measurements), too frequent measurements may cause injury to the patient due to interference with blood flow.


- Measuring blood pressure 1 hour after eating
- Do not use the mobile phone near the device.
- In patients with arrhythmia, measurement results may not be accurate.
- Do not keep the cuff inflated for a long time.
- Note: Do not diagnose by measurement, follow your doctor's instructions.
- Declaration: The tensiometer may not meet specifications if not stored within the required temperature and humidity ranges.
- Do not interconnect this appliance with other appliances not described in the operating instructions.
- In patients with arrhythmia, arterial sclerosis, poor perfusion, diabetes, pregnancy, pre-eclampsia, renal disease, patient movement, tremor, shivering, measurement results may not be accurate.

7. INSTRUCTIONS FOR USE


START MEASURING

Remain seated for 5 minutes before measurement

1. Press the [START | STOP] button to start the measurement after the batteries have been installed. Then the monitor starts pressurizing.

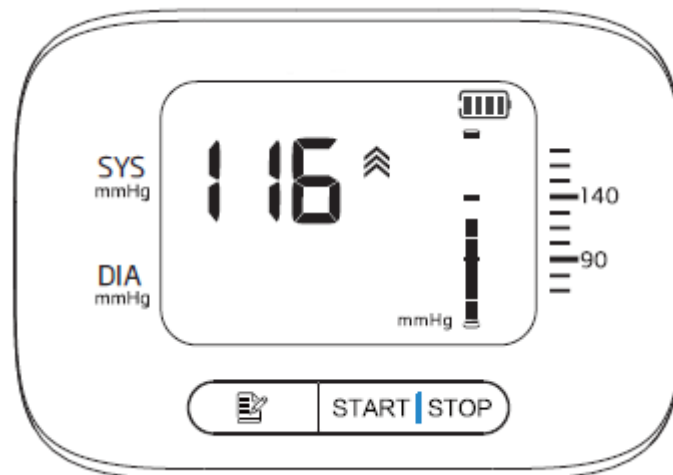2. After the measurement is finished, the monitor will display the SYS (systolic pressure), DIA (diastolic pressure) and pulse rate, then deflates automatically. It will also turn off automatically in 3 minutes without any operation.

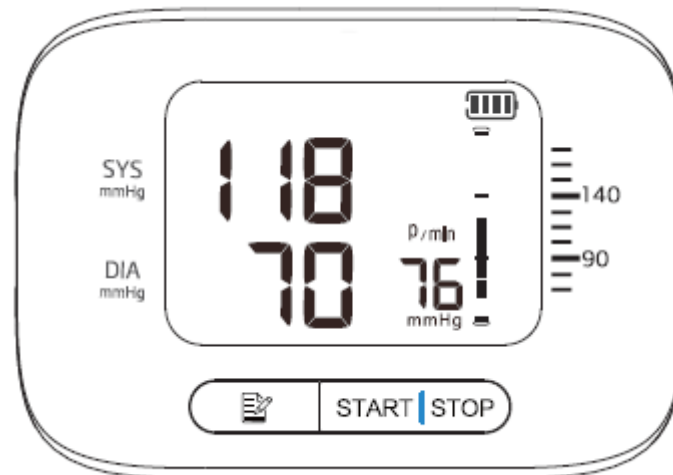- If unexpected readings are obtained, please measure again.
- We suggest that the contact time with the sleeve should not exceed 10 minutes. The maximum surface temperature of the cuff is 42,3 °C.
- If you feel any discomfort, remove the device immediately.

3. Turn off the power: Press the [START | STOP] button to turn off the power. However, if you do not turn it off, it will turn off automatically in 3 minutes without any operation.

Note:

- Do not move and remain silent until the measurement is complete.
- Keep the monitor and the heart at the same level.
- Reset if the cuff loosened during measurement.
- Press the [START | STOP] button to stop the measurement. If it does not work, remove the cuff.
- Warning: The maximum inflation pressure is 300mmHg/40kPa.
- Do not maintain the inflated state for a long time to avoid damage.
- The time interval between two measurements shall be at least 5 minutes.

MEMORY FUNCTION

The monitor will store measurement data automatically (including blood pressure and pulse), and the upper limit of records is 99.

1. Press the "📄" button to check the records.
2. Press the "📄" button for the first time to display the average value of the last 3 measurements.

3. Press the "📄" button again to display the 1st memory group, the serial number is displayed from "1" to "99". "1" is the last group and "99" is the oldest.

>Data 99 can be replaced by 98 when the memory capacity is full.

4. Read the recorded data by pressing the "📄" button according to the sequence: "1", "2" "99" (max). It then returns to the first one.

5. Press and hold the "📄" button to search the data quickly.

6. Press the "🔌" button to turn off the monitor.

DELETING RECORDED DATA

1. Press the [START | STOP] and "📄" buttons until the display shows the image, which means that the recorded data has been erased.

8. MAINTENANCE AND CLEANING

Please observe the following guidelines to protect the device and ensure accurate measurements:

- Store the blood pressure monitor and accessories properly after use.
- Do not leave the blood pressure monitor and accessories exposed to heat, moisture, dust or sunlight.
- The sleeve is equipped with an internal airbag, use it with care and do not bend, pull or twist it.
- Warning: Do not disassemble, repair or modify the appliance without authorisation.
- Do not perform maintenance while the appliance is in use.
- If used by many people, clean the device with a soft cloth dampened with 75% ethanol, but do not allow the ethanol to get into the tensiometer and cuff.
- If used by one person, clean the device with a soft, dry cloth or a soft cloth dampened with a small amount of water, but do not allow water to get into the blood pressure monitor and cuff.
- Do not clean the appliance when it is connected to the mains.

9. SYMBOLS RELATING TO PRODUCT SAFETY REQUIREMENTS AND THEIR MEANING

| | | | |
|---|---|---|--|
|  | Caution |  | Type BF applied part |
|  | Manufacturer |  | Date of manufacture |
|  | Authorised representative in the European Community/European Union |  | Importer |
|  | Waste Electrical and Electronic Equipment (WEEE), indicating that separate collection is necessary for waste electrical and electronic equipment (WEEE) | | |
| IP22 | Indicates the degree of protection against harmful ingress of water and particles according to IEC 60529 | | |
|  | CE marking and notified body number | | |
|  | Keep dry |  | Refer to instruction manual |
|  | This way up |  | Batch code |
|  | Fragile/handle with care |  | Unique device identifier |
| DC | Direct current |  | Unsafe MR, which identifies an element that poses unacceptable risks to the patient, medical staff or others within the MR environment |
|  | Direct current | | |
|  | Serial number |  | Medical devices |

10. CONDITIONS OF GUARANTEE

This product is covered by a 3-year warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with our VITALPRESS digital wrist blood pressure monitor! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com. We appreciate your comments and suggestions for further improvement.

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CO., LTD.
GRUPO R. QUERALTO S.A.