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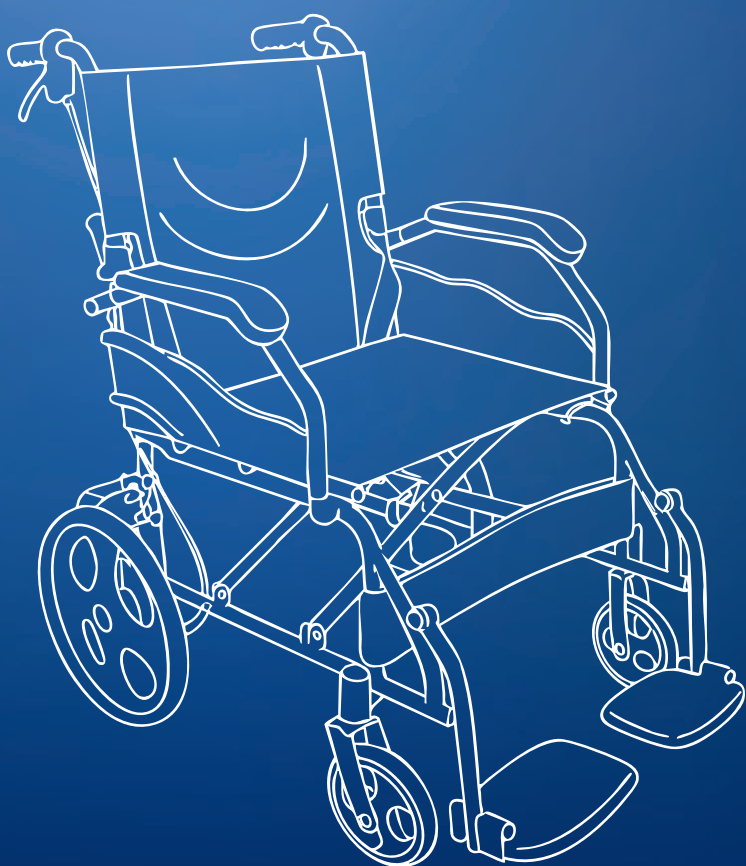
IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the FARAÓN wheelchair! In this instruction manual you will find all the information you need to assemble, use and maintain the wheelchair safely.

This wheelchair combines strength, durability and ergonomic design, making it an excellent choice for both everyday use and clinical settings.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

1. **WARNING!** Never leave a person unattended while using the wheelchair, especially if the user has limited mobility or balance problems.
2. **WARNING!** Use the wheelchair only for its intended purpose.
3. **WARNING!** Activate the brakes when the wheelchair is in stationary use to avoid unexpected movements. Check the brakes regularly to ensure that they are working properly.
4. **WARNING!** Avoid sudden movements or extreme tilting while in the wheelchair to prevent falls or tipping.
5. **WARNING!** Use only original spare parts and accessories to ensure compatibility and safety.
6. **WARNING!** Before using this product, please check that all parts are correctly assembled.
7. **WARNING!** The user's weight must not exceed the maximum weight capacity (220,46 lb).
8. **WARNING!** Avoid exposure to sunlight and store in a cool, dry and ventilated place.
9. **WARNING!** Before pushing the product, make sure that the user's feet are placed on the footrests and that the brakes are released.

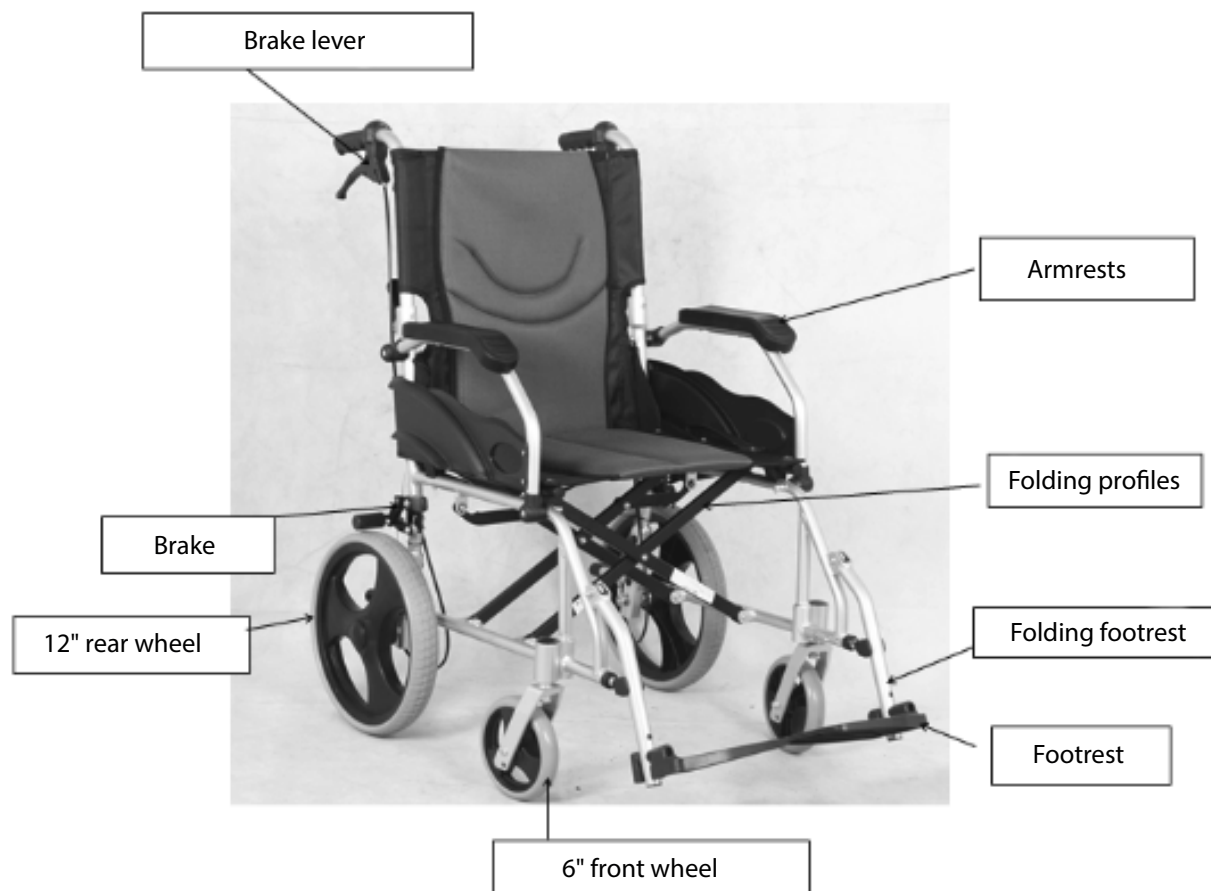
2. PURPOSE OF THE PRODUCT

- **Facilitating mobility:** Enabling autonomous or assisted movement for people with reduced mobility.
- **Provide support and comfort:** Offer a safe and comfortable posture during use, thanks to its ergonomic and resistant structure.
- **To provide independence to the user:** To promote autonomy in daily activities both at home and outdoors, increasing the quality of life.
- **Withstand continuous use:** Its steel structure is designed to withstand intensive use in a variety of environments, ensuring durability and reliability.
- **Adapt to different environments:** Be functional in hospitals, rehabilitation centres, homes and public spaces.

3. PRODUCT FEATURES

PRODUCT NAME	Steel wheelchair
MODEL	FARAÓN
MODEL REFERENCE	A-00461/09-AZ
MAXIMUM SUPPORTED WEIGHT	220,46 lb
NET PRODUCT WEIGHT	27,33 lb
PRODUCT DIMENSIONS	24,60"* 40,94"*34,25"
MATERIALS	Flame retardant Oxford fabric and steel
OTHER CHARACTERISTICS	Seat width: 18,11" Rear wheel inches: 12"

4. PARTS AND COMPONENTS

5. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

- Restricted or limited mobility (e.g. legs and/or hips).
- Balance problems in standing or walking due to permanent transient disabilities.
- The weight of which does not exceed 220,46 lb.

CONTRAINDICATIONS

- Users with open wounds, ulcers or infections on areas of the body that will come into contact with the chair, which could worsen the condition.
- Patients whose weight exceeds the maximum capacity of the chair, which could compromise the structure and safety of the device.
- People who cannot sit upright on their own and cannot be safely secured in the chair, which increases the risk of falls.
- Situations where the wheelchair could be used incorrectly, such as use on unsuitable or dangerous surfaces.

6. INSTRUCTIONS FOR USE

The FARAÓN wheelchair comes fully assembled and folded. Below are the steps for folding and unfolding the wheelchair and some instructions to take into account in the process.

FOLDING OF THE CHAIR:

1. Lift the footrests upwards from horizontal to vertical position.

2. Then lift the folding footrest frame upwards to a horizontal position in relation to the floor (90° turn).

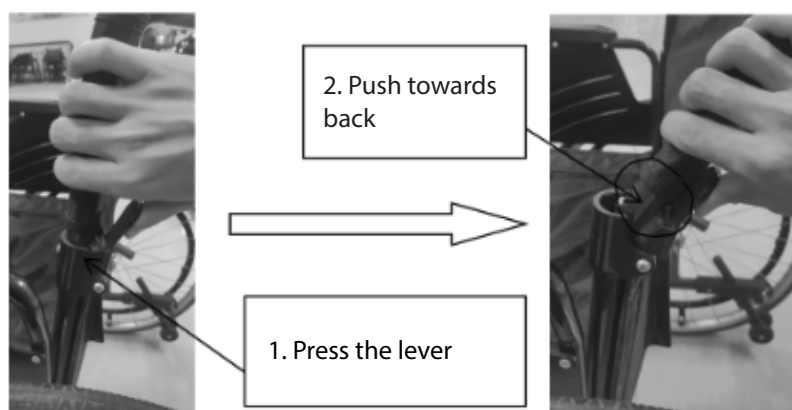
3. For the next step, gently push the centre of the seat downwards and then push the armrests inwards to complete the gluing of the seat. Refer to the pictures below.

4. To complete the folding of the footrest frame, push up on the footrests until they engage with the folded chair frame as shown in the following pictures.

5. Next, the backrest of the chair is folded. To do this, use the folding handle on the back of the chair. The steps for folding the backrest are as follows:

6. The folding of the chair would be completed.

UNFOLDED FROM THE CHAIR:

To deploy the wheelchair, carry out the process described above in reverse:

1. Correct positioning of the backrest.
2. Lowering of the footrest structure.
3. Push down on the armrests to return the seat to its original position and listen for a "click" to ensure that the seat is correctly fixed.
4. Place the footrests in a horizontal position.

LENGTH ADJUSTMENT OF THE FOLDING FOOTREST STRUCTURE:

To adjust the length of the footrest, press the button marked on the image to adjust the length of the footrest accordingly. There are three possible lengths to adjust.

**USE OF THE BRAKE:**

- Lift the lever on the right-hand side to brake the wheels.
- Push down on it to loosen and release the wheels from the brake.

7. MAINTENANCE AND CLEANING

- Use a damp cloth with lukewarm water.
- Avoid the use of aggressive chemicals that may damage the materials.
- Visually check the chair for any damage, wear and tear or loose parts before each use.
- Make sure that the wheels move freely and the brakes work properly.

8. PROBLEM SOLVING

Problem: The chair does not fold correctly.

Cause: Folding handle obstructed or blocked.

Solution: Check that there are no objects trapped in the mechanism.

Problem: Wheels do not turn smoothly.

Cause: Dirt accumulation or blockages in place.

Solution: Clean the wheel axle and make sure the wheel locks are released.

Problem: Brakes do not apply correctly.

Cause: Brake wear or poor adjustment.

Solution: Check brake cable tension and contact service if necessary.

Problem: The seat or backrest shows signs of wear or damage.

Solution: Regularly inspect the seat and backrest for damage. If significant damage is found, discontinue use of the chair until the seat or backrest has been repaired or replaced. Contact service if necessary.

9. CONDITIONS OF GUARANTEE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with this FARAÓN wheelchair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com**. We appreciate your comments and suggestions for further improvement.

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FOSHAN KAIYANG MEDICAL EQUIPMENT CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.