MANUAL DE INSTRUCCIONES

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TROYA/TROYA PLUS/TROYA PLUS ULTRA

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SW
Instrukcja obsługi	PL
Brugsanvisning	DA

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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the TROYA/TROYA PLUS/TROYA PLUS ULTRA power wheelchair! In this instruction manual you will find all the information you need to assemble, use and maintain the wheelchair safely.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

1. **WARNING!** Always use the seat belt to avoid falling or slipping during movement.
2. **WARNING!** Do not exceed the maximum permissible weight (264,55 lb).
3. **WARNING!** Avoid slopes, kerbs and steps exceeding the safety specifications, as they may cause rollover.
4. **WARNING!** Do not drive on dangerous surfaces such as deep water, loose sand, unstable gravel or slippery areas.
5. **WARNING!** Do not use the wheelchair near fire, oxygen, petrol or explosive atmospheres, as this may cause fire or explosion.
6. **WARNING!** Do not transport more than one person or allow passengers, the equipment is designed for one user only.
7. **WARNING!** Switch off the wheelchair before getting in or out of it, to prevent unexpected movements.
8. **WARNING!** Do not tamper with the electrical system and batteries, repairs must only be carried out by authorised personnel.
9. **WARNING!** Use only the charger recommended by the manufacturer and charge in a dry and ventilated environment.
10. **WARNING!** Do not expose the chair to heavy rain, immersion in water or pressure washing to avoid electrical faults.

2. PRODUCT FEATURES

PRODUCT NAME	Electric wheelchair
MODEL	TROYA TROYA PLUS TROYA PLUS ULTRA
MODEL REFERENCE	TROYA (A-00480/05) TROYA PLUS (A-00480/06) TROYA PLUS ULTRA (A-00480/07)
MAXIMUM SUPPORTED WEIGHT	264,55 lb
NET PRODUCT WEIGHT	TROYA (57,32 lb) TROYA PLUS (61,72 lb) TROYA PLUS ULTRA (66,13 lb)
PRODUCT DIMENSIONS	40,94" (length)*25,19" (width)* 35,82" to 41,73" (height)

3. PARTS AND COMPONENTS

1 Wheelchair

2 Accessories bag

Charger

Tools

Anti-tipper

Control

3 Model TROYA (1 Battery, 17km autonomy)

Model TROYA PLUS (2 Batteries, 34 km range)

Model TROYA PLUS ULTRA (3 Batteries, 51 km autonomy)

4. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

- Power wheelchairs are intended for people with mobility difficulties, including people with disabilities, patients and elderly or physically weak people as mobility aids.
- Weight capacity: 264,55 lb.

CONTRAINDICATIONS

- Not recommended for people who are too weak to sit up.
- People who have difficulty in driving a wheelchair due to various disabilities.
- If the user's weight exceeds the maximum load capacity clearly indicated on the wheelchair.

5. HOW TO FOLD OR UNFOLD THE CHAIR

METHOD 1. To open the wheelchair:

1. Stand behind the wheelchair, hold the handlebars with your hands, tilt the wheelchair slowly forward and lift it at the same time.
2. Continue to lift until you hear a click when the middle spring pin is inserted into the fixing hole.
3. The wheelchair can be easily unfolded.

METHOD 2. To open the wheelchair:


1. Stand at the side of the wheelchair, hold the handlebars with one hand and push the seat down with the other hand at the same time.
2. Continue to push the seat down until you hear a click when the spring pin in the centre is inserted into the fixing hole.
3. The wheelchair can be easily unfolded.

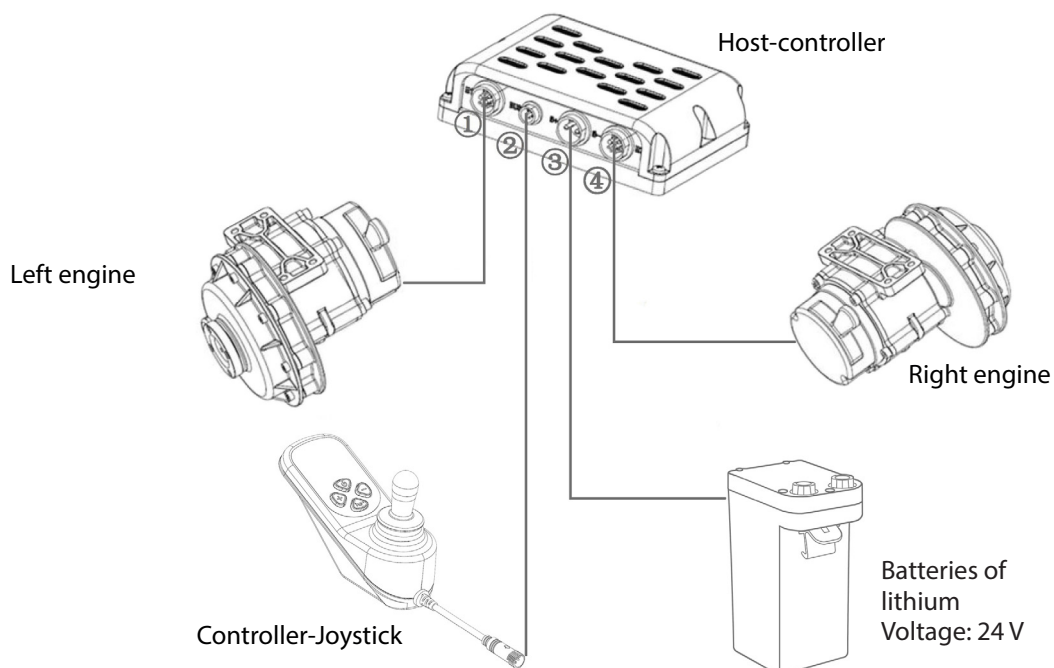METHOD 3. To fold the wheelchair:

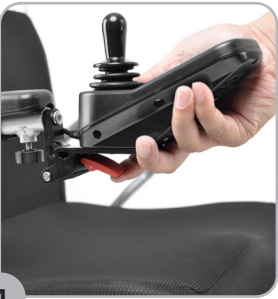
1. Stand to the side of the wheelchair, pull the knob with one hand and push the handlebar towards the seat with the other hand so that the spring pin deflects from the fixing hole, which is for the next step: folding the wheelchair easily.
2. Continue standing at the side of the wheelchair, hold the handlebars with one hand, grasp the front of the seat frame with the other hand and pull it towards the backrest until the wheelchair is fully folded.
3. Lift the lever under the controller to fold the wheelchair until it is completely folded.

6. ELECTRICAL DIAGRAM

7. HOW TO INSTALL AND CONNECT THE JOYSTICK


 <p>1</p>	 <p>2</p>	 <p>3</p>
<p>Remove the controller from the accessory bag and insert it into the tube of the arm bracket.</p>	<p>Turn the knob clockwise to lock the controller bracket.</p>	<p>Connect the controller connector and the communication power cable.</p>
 <p>4</p>	 <p>5</p>	 <p>6</p>
<p>Lift the lever.</p>	<p>The controller bracket can be easily folded down.</p>	<p>After returning the lever to its original position, lift the knob bracket until you hear the spring pin inserted in the hole click and the knob bracket is locked.</p>

8. HOW TO INSTALL AND REMOVE BATTERIES

Install the lithium batteries:

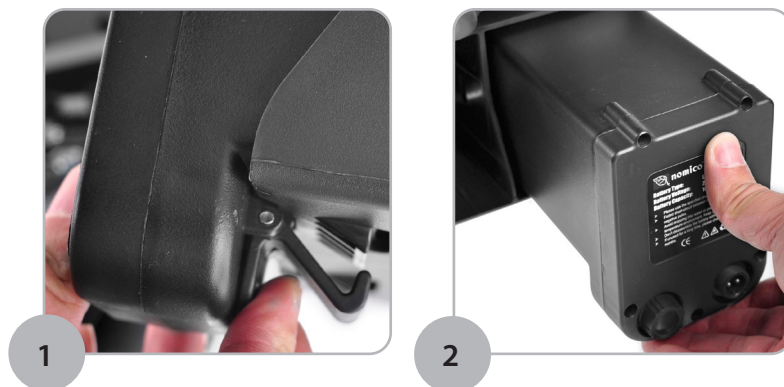
Remove the lithium batteries:


9. HOW TO DRIVE AND CONTROL THE WHEELCHAIR

9.1. Change the driving mode

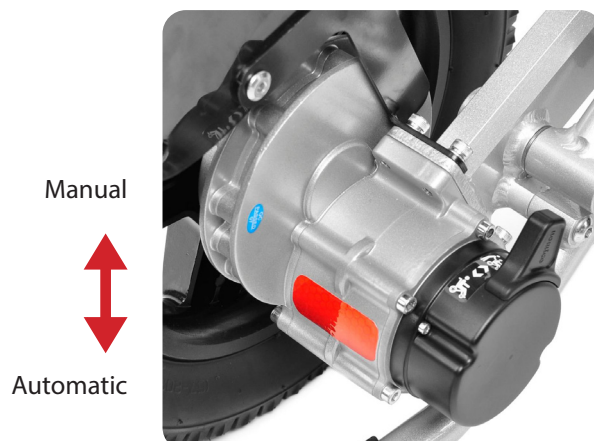The manual mode (M) allows the user to move the wheelchair without using the battery, i.e. by pushing the wheelchair. While the automatic mode (D) will allow you to both drive the wheelchair and keep it stationary when you want to park it.

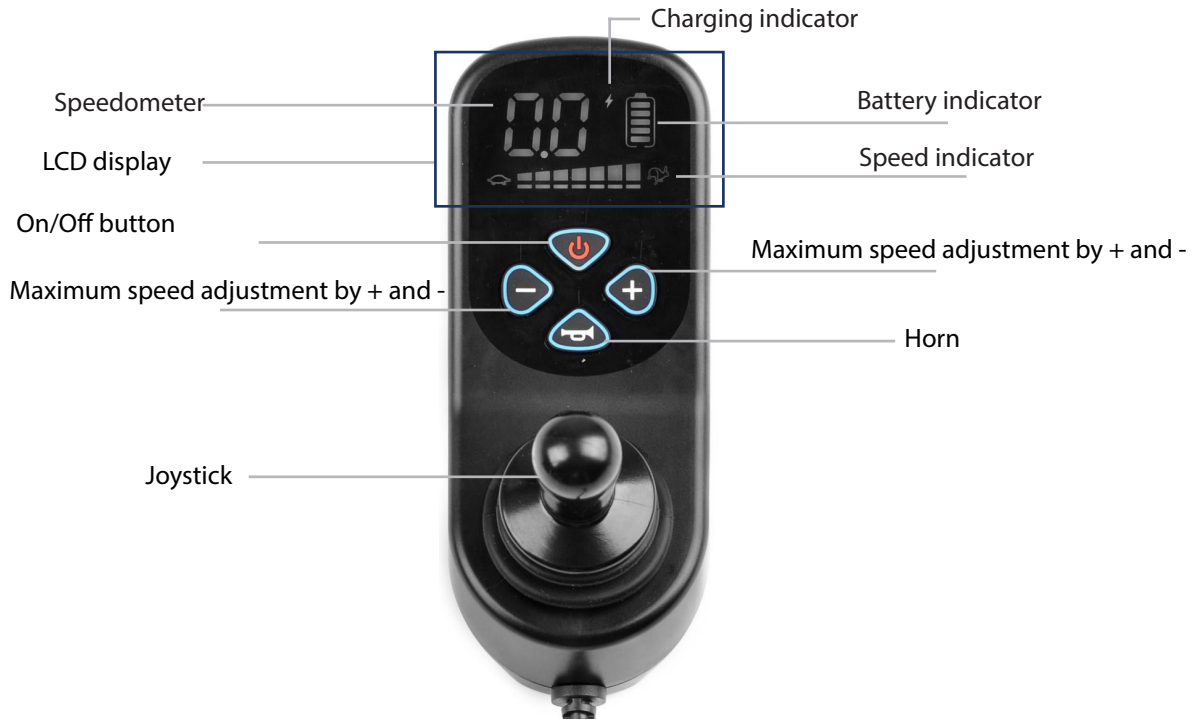
9.2. Description of the standard controller buttons

10. MORE FUNCTIONS

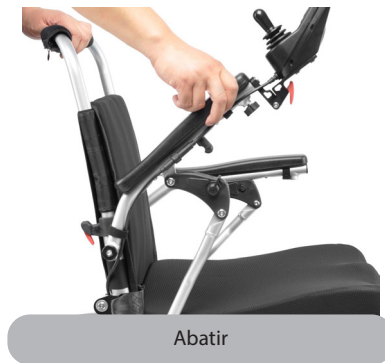10.1. Folding armrests:

10.2. Folding footrests:

10.3. Height adjustable backrest:

11. CHARGING OF LITHIUM BATTERIES

Charging time is usually 2 to 8 hours.

Method 1. To load via the joystick:

1

Connect the charger

2

Connect the joystick


3

Connect to the mains socket


Method 2. To charge by cable (lithium batteries are charged directly) (Cable not included, sold separately):

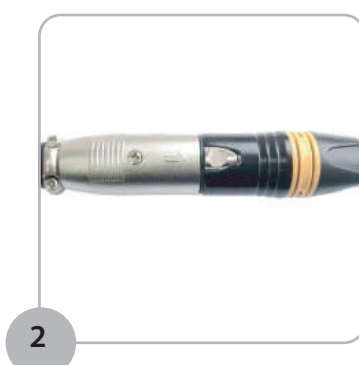1

2

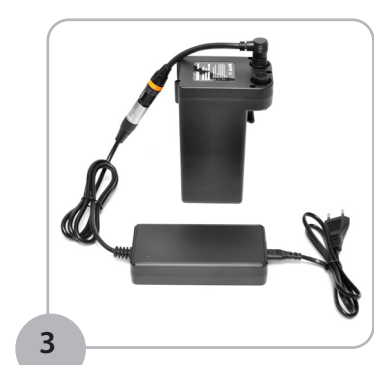3

CAUTION:

1. For indoor use only, do not expose to rain.
2. Disconnect the power supply before removing the battery charger.
3. Place the charger in a well-ventilated area during charging.
4. Use the charger only with 24V lithium batteries.
5. To ensure the product's service life, whether or not it is used on a recurring basis, we recommend charging the batteries every 3 to 4 days.

12. TECHNICAL CHARACTERISTICS

PARAMETERS	
Engine	220W*2, DC 24V
Maximum speed	8.5 km/h
Braking distance	≤ 39,37"
Max. slope capacity	15°
Turning radius	33,46"
Rear wheel	12" / Max. pressure 2,5 bar
Front wheel	8"
Lithium battery	24V10Ah/pc
Net weight of battery	3,74 lb
Autonomy	20 km/1 battery-40 km/2 batteries-60 km/ 3 batteries
CD changer output	24V/24
AC charger input	100-240V, 50/60Hz

MEASUREMENTS	
Seat width	17,71"
Seat depth	17,32"
Seat height	19,68"
Wide backrest	17,32"
Backrest width	18,50"
Distance from seat to armrest	8,26"
Ground clearance	2,55"
Wheelbase	23,62"
Overall length	40,94"
Total width	25,19"
Total height	35,82" - 41,73" (height adjustable)
Dimensions folded	25,19"* 13,38"* 29,13"

13. MAINTENANCE AND CLEANING

- Regularly check wheels, tyres, brakes and joystick, ensuring that everything is working properly before use.
- Take care of the batteries by keeping them charged, checking the connections and preventing them from discharging completely.
- Clean the structure with a damp cloth and neutral soap, drying thoroughly afterwards; do not use pressurised water or immerse the chair.

14. SOLUTIONS FOR COMMON FAILURES

SY-C20 CONTROLLER

When the fault code appears on the controller display, it indicates that the power wheelchair is in an abnormal condition. The meaning of the display and the recommended method of elimination are shown in the following table.

FAULT CODE	DESCRIPTION OF FAILURE AND WARNING	ELIMINATION METHOD
C1	Communication failure	Check if the power cable is intact and the connector is correctly connected
C2	The joystick is not in the home position	Turn off the power, release the joystick back to the centre position, then restart
C4 + 	Left brake or controller failure	Check if the left motor crank is switched to the drive state, the power cable is intact and the connector is properly connected, then restart. If the problem is still not solved, connect the left and right interchangeable motors, if C4 and  appear, it means that the left brake is defective; if C4 and  still appear, it means that the controller is defective
C4 + 	Right brake or controller failure	Check if the crank of the right motor is switched to the drive state, the power cable is intact and the connector is properly connected, then restart. If the problem is still not solved, switch on the left and right motors indistinctly, if C4 and  appear, it means that the right brake is defective; if C4 and  still appear, it means that the controller is defective

C5	Overload warning	Switch off the power and switch the engine to the manual state, check if the engine can turn smoothly, if so, switch back to the driving state and restart
C6	Ignition failure	Disconnect power and restart
C7	Overvoltage warning	Disconnect the power, reconnect the battery cable and restart. Try replacing the controller or battery if the problem is still not resolved
C8	Low voltage warning	The battery is low, please charge it in time. If the battery is fully charged and still low voltage alarm, it means that the controller is defective
L1 / R1	Left or right motor overcurrent warning	Disconnect the power supply and put the motor in manual state, check if the motor can rotate smoothly, if not, disconnect the corresponding motor connector and the drive controller and check again, it means the motor is short-circuited if it still cannot rotate smoothly, on the contrary, it means the controller is defective if it can rotate smoothly
L2 / R2	Controller failure	Contact after-sales service
L3 / R3	Circuit overheating warning	Disconnect the power and wait for it to cool naturally, then restart to check for a fault, if the fault still exists, contact after-sales service to replace the controller

<p>L6 / L7 / L8</p>	<p>Left engine or controller failure</p>	<p>Check if the power cable of the left motor is intact and the connector is properly connected. If the problem persists after the check, connect the left and right motors indistinctly and restart, it means the left motor is faulty if R6 / R7 / R8 is displayed, it means the controller is faulty if L6 / L7 / L8 is still displayed</p>
<p>R6 / R7 / R8</p>	<p>Right engine or controller failure</p>	<p>Check if the power cable of the right motor is intact and the connector is properly connected. If the problem persists after the check, connect the left and right motors indistinctly and restart, it means the right motor is faulty if L6 / L7 / L8 is displayed, it means the controller is faulty if R6 / R7 / R8 is still displayed</p>

NOTICE:

Switch off the wheelchair, leave it for a few minutes and restart it. If the fault persists, you should switch off the wheelchair and contact your dealer. Make a note of the information on the control panel display and report it to your dealer.

Do not use the wheelchair until the problem has been resolved or you have received further instructions from your service provider.

15. CONDITIONS OF GUARANTEE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with your TROYA/TROYA PLUS/TROYA PLUS ULTRA power wheelchair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at **clientes@mobiclinic.com**. We appreciate your comments and suggestions for further improvement.

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SUZHOU IDEAL MEDICAL CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.