MANUAL DE INSTRUCCIONES

SILLA DE RUEDAS

KEFRÉN

Manual de instrucciones	ES
Instruction manual	EN
Manuel D'Utilisation	FR
Manuale D'Istruzioni	IT
Anweisungen	DE
Instruções	PT
Gebruiksaanwijzing	NL
Bruksanvisning	SW
Instrukcja obsługi	PL
Brugsanvisning	DA

ES - Lea las instrucciones antes de usar el producto.
EN - Please read instructions before use.
FR - Lisez les instructions avant d'utiliser le produit.
IT - Leggere le istruzioni prima di usare il prodotto.
DE - Lesen Sie die Gebrauchsanweisung, bevor Sie das Produkt verwenden.
PT - Leia as instruções antes de utilizar o produto.
NL - Lees de instructies voordat u het product gebruikt.
SW - Läs instruktionerna innan du använder produkten.
PL - Läs instruktionerna innan du använder produkten.
DA - Læs vejledningen, før du bruger produktet.

IMPORTANTE

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IMPORTANT

Keep these instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing our KEFRÉN steel wheelchair! In this manual, you will find all the information you need to assemble, use and maintain this product correctly and safely.

Before you begin, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. This product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

1. **WARNING!** Never leave a person unattended while using the wheelchair, especially if the user has limited mobility or balance issues.
2. **WARNING!** Use the wheelchair only for its intended purpose.
3. **WARNING!** Apply the brakes when the chair is stationary to prevent unexpected movement. Check regularly that they are working properly.
4. **WARNING!** Avoid sudden movements or extreme tilts while in the wheelchair to prevent falls or tipping.
5. **WARNING!** Only use original spare parts and accessories to ensure compatibility and safety.
6. **WARNING!** Before using this product, please check that all parts are correctly assembled.
7. **WARNING!** The user's weight must not exceed the maximum weight capacity (220,46 lb).
8. **WARNING!** Avoid exposure to sunlight and store in a cool, dry, well-ventilated place.
9. **WARNING!** Before pushing the product, ensure that the user's feet are placed on the footrests and that the brakes are unlocked.

2. PRODUCT PURPOSE

1. **Facilitating mobility:** Enabling assisted travel for people with reduced mobility.
2. **Provide support and comfort:** Offer a secure and comfortable posture during use, thanks to its ergonomic and sturdy structure.
3. **Withstand continuous use:** Its steel structure is designed to withstand intensive use in different environments, ensuring durability and reliability.
4. **Adapting to different environments:** Being functional in hospitals, rehabilitation centres, homes, and public spaces.

3. PRODUCT FEATURES

PRODUCT NAME	Steel wheelchair with 8" rear wheel
MODEL	KEFRÉN
MODEL REFERENCE	A-00462/07-GR
MAXIMUM WEIGHT SUPPORTED	220,46 lb
NET WEIGHT OF THE PRODUCT	26,01 lb
PRODUCT DIMENSIONS	24,60"* 40,94"* 34,25"
MATERIAL	Fire-resistant Oxford fabric and steel
OTHER FEATURES	<ul style="list-style-type: none"> • Rear wheels: 8" • Front wheels: 6" • Folded width: 12,59" • Seat height from floor: 16,92" • Seat depth: 14,17" • Seat width: 18,11" • Brakes: United (act on both wheels simultaneously)

4. PARTS AND COMPONENTS

5. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

This wheelchair is suitable for people with:

- Restricted or limited mobility (e.g., legs and/or hips).
- Balance problems when standing up or walking due to temporary or permanent disabilities.
- Whose weight does not exceed 220,46 lb.

CONTRAINDICATIONS

This wheelchair is not suitable in the following cases:

- Users with open wounds, ulcers, or infections in areas of the body that will come into contact with the chair, which could worsen the condition.
- Patients whose weight exceeds the maximum capacity of the chair, which could compromise the structure and safety of the device.

- People who cannot sit upright by themselves and cannot be securely fastened in the chair, which increases the risk of falls.
- Situations in which the wheelchair could be used incorrectly, such as on unsuitable or dangerous surfaces

6. INSTRUCTIONS FOR USE

The Kefrén wheelchair comes fully assembled and folded. Below are the steps for folding and unfolding the chair and some instructions to keep in mind during the process.

FOLDING THE CHAIR

1. Raise the footrests upwards from the horizontal to the vertical position.

2. Next, lift the folding footrest structure upwards until it is horizontal to the floor (90° turn).

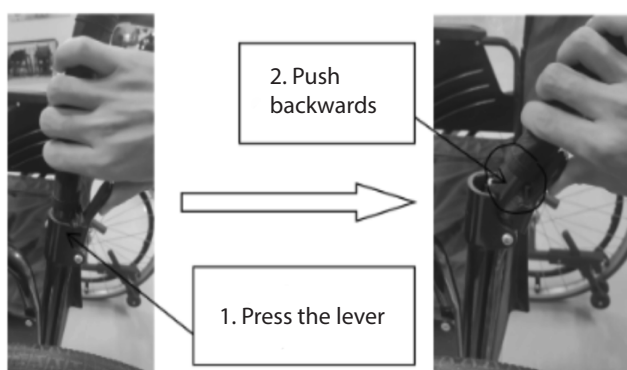
- For the next step, gently push down on the centre of the seat and then push the armrests inwards to complete the seat attachment. Take a look at the following images.

- To complete the folding of the footrest structure, push it upwards until it clicks into place with the folded chair structure, as shown in the following images.

- Next, fold down the backrest of the chair. To do this, use the folding handle located at the rear of the chair. The steps for folding down the backrest are as follows:

The chair will now be fully folded.

7. MAINTENANCE AND CLEANING

REGULAR MAINTENANCE:

- Visually inspect the chair for any damage, wear or loose parts before each use.
- Ensure that the wheels move freely and the brakes function correctly.
- Apply lubricating oil to moving parts weekly to prevent rough operation.
- Store the wheelchair in a dry, ventilated, stable and non-corrosive environment.

CLEANING:

- Use a cloth dampened with warm water
- Avoid using harsh chemicals that could damage the materials.
- After using the wheelchair, wipe the surface with a soft, dry cloth to remove moisture and dirt, thus preventing rusting.
- The seat and backrest should be kept clean regularly to prevent skin infections.

8. PROBLEM SOLVING

Problem: The chair does not fold properly.

- Possible cause: Folding handle obstructed or blocked.
- Solution: Check that there are no objects trapped in the mechanism.

Problem: The wheels do not turn smoothly.

- Possible cause: Accumulation of dirt or blockages.
- Solution: Clean the wheel axle and ensure that the locks are released.

Problem: The brakes do not engage properly.

- Possible cause: Brake wear or poor adjustment.
- Solution: Check the tension of the brake cable and contact technical support if necessary.

Problem: The seat or backrest shows signs of wear or damage.

- Solution: Regularly inspect the seat and backrest for damage. If you find any significant damage, stop using the chair until the seat or backrest has been repaired or replaced. Contact technical support if necessary.

9. WARRANTY CONDITIONS

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for choosing Mobiclinic for this KEFRÉN steel wheelchair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or require further assistance, please do not hesitate to contact us via our customer service department at **clientes@mobiclinic.com**. We appreciate your comments and suggestions so that we can continue to improve.

Thank you for being part of Mobiclinic!

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FOSHAN KAIYANG MEDICAL EQUIPMENT CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTO S.A.