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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product.

Thank you for choosing the NAPOLI wheelchair! In this manual you will find all the information you need to assemble, use and maintain this product correctly and safely.

Our manual wheelchair has been designed to offer mobility, safety and comfort to people with difficulties to move independently. Manufactured with resistant and durable materials, it can be used indoors, facilitating daily movements with total confidence.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

- 1. WARNING!** Do not use the product if the user has conditions that can be aggravated by contact with PVC materials.
- 2. WARNING!** Use the wheelchair only for its intended purpose. Do not use it for carrying heavy objects or as a support for standing or sitting.
- 3. WARNING!** Always apply the brakes when the wheelchair is in stationary use to avoid unexpected movements. Regularly check the brakes for proper function.
- 4. WARNING!** Avoid sudden movements or extreme tilting to prevent falling or tipping.
- 5. WARNING!** Use only original spare parts and accessories to ensure compatibility and safety.
- 6. WARNING!** Strictly follow the assembly instructions, steps and assembly methods.
- 7. WARNING!** If you encounter mechanical errors or abnormal operation of the product, please contact the technical team.
- 8. WARNING!** The user's weight must not exceed the maximum weight capacity (220,46 lb).
- 9. WARNING!** This product is suitable for indoor use.

10. WARNING! Avoid exposure to sunlight and store in a cool, dry and ventilated place.

11. WARNING! Before pushing the product, make sure that the user's feet are placed on the footplate and that the brakes are released

12. WARNING! Do not stand on the footplate to prevent falls and possible breakage of the wheelchair.

2. PURPOSE OF THE PRODUCT

1. Facilitating daily mobility: Helps people with reduced mobility to move around safely and comfortably, enabling them to carry out everyday activities.
2. Providing assistance to people with limited mobility: Designed for elderly people, patients undergoing rehabilitation, or disabled users who need support in their transfers.
3. Ensuring comfort and safety: Designed with fixed armrests, folding backrest, removable footrests and solid castors to ensure stability during use.

3. BENEFITS AND CONSIDERATIONS

The Napoli manual wheelchair has been developed to offer a safe and comfortable solution for people with mobility difficulties. Its folding frame makes it easy to store and transport, while the high quality materials ensure durability and strength.

Before use, it is essential to check that all parts are correctly installed and that the brakes are working properly. The recommended weight capacity (100 kg) should not be exceeded to ensure user safety. In addition, regular maintenance is recommended to prolong the life of the product and ensure its proper functioning.

4. PRODUCT FEATURES

PRODUCT NAME	Wheelchair
MODEL	NAPOLI
MODEL REFERENCE	A-00039/45
MAXIMUM SUPPORTED WEIGHT	220,46 lb
NET PRODUCT WEIGHT	24,47 lb
PRODUCT DIMENSIONS	36,61"*22,83"*37,79"

COLOUR	Blue and black
MATERIAL	Steel
OTHER CHARACTERISTICS	<ul style="list-style-type: none"> • Slope performance: 8° • Static stability (front and rear tilt): 10° • Static stability (lateral tilt): 15° • Rolling offset: 13,77" • Minimum turning radius: 33,46" • Minimum width for change of direction: 59,05" • Seat width: 16,14" • Height from seat to floor: 19,68" • Seat depth: 15,74" • Backrest height: 16,92" • Front wheel size: 8" • Rear wheel size: 8" • Welded, high-strength tubular steel structure • Folding wheelchair • Removable footrests • Fixed armrests • Folding backrest

5. PARTS AND COMPONENTS

6. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

This wheelchair is suitable for people with:

- People with injuries or diseases of the lower limbs, or injuries or diseases of the nervous system: Such as those with lower limb disabilities, hemiplegia and paraplegia below the chest. Wheelchairs can help them regain some freedom of movement.
- Patients with severe heart disease: For patients with severe heart disease or those suffering from total heart failure due to other illnesses, wheelchairs offer a way to get around without requiring physical exertion.
- Patients with central nervous system diseases: For patients with central nervous system diseases who are at risk when walking independently, as well as for people who have difficulty walking and are prone to accidents, wheelchairs provide a guarantee of safety.
- Elderly and frail people: For elderly people who are frail and have difficulty walking steadily, wheelchairs can assist them in their daily activities.

CONTRAINDICATIONS

This bath wheelchair is not suitable for the following cases:

- Patients with severe pressure ulcers of the buttocks or those with unhealed pelvic fractures: The use of a wheelchair may aggravate the injuries of such patients and may not be conducive to their recovery.

7. ASSEMBLY INSTRUCTIONS

STEP 1 - Unfold the chair

- Place the chair on a flat surface, with the wheels touching the ground (figure 1).
- Grasp the padded armrests (part 8), one with each hand.
- Pull sideways (left and right) at the same time. This will open the frame (part 4).
- Continue to spread until the seat is fully extended and firm as shown in figure 2.

Figure 1.

Figure 2.

STEP 2 - Install the removable footrests

- Take one of the footrests (part 10).
- Locate the metal hole at the bottom of the side frame of the chair.
- Align the upper pin of the footrest with the hole in the footrest
- Insert the footrest and turn it inwards until it locks, it should be as shown in figure 3.

Figure 3.

STEP 3 - Attach the foot strap

- Pass the strap under or through the two footrests.
- Fasten the Velcro firmly to maintain a comfortable but secure fit.

Figure 4.

STEP 4 - Raise the ergonomic handles with brake

- They come pre-installed.
- You have to unfold them (Part 1) upwards, leaving the structure in its functional position.
- To release the brakes (Part 2), gently press the red lever inwards.
- This releases the brake pressure. It is a pull brake mechanism: when squeezed, it is deactivated.
- It should look as shown in figure 6.

Figure 5.

Figure 6.

STEP 5 - Activating the brake system without using the brake lever

- It has a manual brake pedal: It is located on the rear wheels.
- Gently lift the pedal upwards with your foot: Do this in a firm but controlled movement. The pedal will rise and release the wheel, indicating that the brake has been correctly disengaged. You can see in figure 8 how the wheel would look when unlocked.

Figure 7.

Figure 8.

This is how it should finally be assembled:

Figure 9.

8. INSTRUCTIONS FOR USE

Ensure that the brakes are applied before the rider sits down. Depress the brake levers located on each rear wheel.

Disengage the brakes: Before you start moving the wheelchair, make sure to disengage the brakes on the rear wheels. Simply repeat the procedure in reverse.

Use of the footrest:

- Before sitting down: Lift the footrest and do not stand on the plates.

Figure 10.

- Once seated: return the footrest to its original shape so that you can use it.

Figure 11.

On rising:

Follow the same procedure to raise the footplate. Do not stand on the footplate as it may tip forward.

9. MAINTENANCE AND CLEANING

Regular maintenance and proper cleaning are important to ensure the durability and proper functioning of the product:

- Visually check the chair for any damage, wear and tear or loose parts before each use.
- Make sure that the wheels move freely and the brakes work properly.
- Check all moving parts, footrests and adjustment mechanisms to ensure that they operate smoothly.
- Use a damp cloth with lukewarm water.
- Avoid the use of aggressive chemicals that may damage the materials.
- When the product is not in use, it should be placed in a dry and ventilated place.

10. PROBLEM SOLVING

1. Difficulty in assembling components:

- **Problem:** Parts do not fit correctly or do not fit according to instructions.
- **Solution:** Make sure you are using the correct parts according to the instructions. If the problem persists, check the illustrations in the manual and make sure all parts are oriented correctly. If you still cannot assemble the chair, contact customer service.

2. Wheels not turning properly:

- **Problem:** Wheels are blocked or do not turn smoothly.
- **Solution:** Inspect wheels for obstructions such as hair, dirt or debris. Clean the wheels and lubricate the axles as necessary. Make sure the brakes are not applied while attempting to move the wheelchair.

3. Brakes not working:

- **Problem:** Brakes do not activate.
- **Solution:** Check the brake adjustment and ensure that the brakes are correctly fitted. Clean off any dirt that may be preventing the brakes from working. If the brakes are worn, consider replacing them.

4. Footrests and armrests loose:

- **Problem:** Footrest and armrests are loose or not held in position.
- **Solution:** Ensure that the footrest and armrest are correctly installed and tightened. Tighten any loose screws or bolts. If the part still does not work properly, consult the manufacturer for replacement parts.

5. Damaged seat or backrest:

- **Problem:** The seat or backrest shows signs of wear or damage.
- **Solution:** Regularly inspect the seat and backrest for damage. If significant damage is found, discontinue use of the chair until the seat or backrest has been repaired or replaced. Contact the manufacturer for replacement parts.

11. CONDITIONS OF GUARANTEE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for trusting Mobiclinic with this NAPOLI wheelchair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com. We appreciate your comments and suggestions for further improvement.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.