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IMPORTANT

Keep the instructions for future reference. Read these instructions before assembling and using this product.

Thank you for choosing the TRAJANO folding walker! In this manual you will find all the information you need to use and maintain this product correctly and safely.

This walker has been designed to give you more independence, comfort and safety during your daily mobility. This product combines a lightweight yet sturdy structure, easy-rolling wheels, ergonomic brakes and a practical seat with backrest, allowing the user to rest when needed.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

1. **WARNING!** The load together with the weight of the user must not exceed the maximum supported weight.
2. **WARNING!** Do not use the walker as a transport chair while in motion. The seat is only designed to rest when the walker is at a complete stop and the brakes are applied.
3. **WARNING!** Do not walk too far away from the walker. Keep your body within the support area of the walker to avoid imbalance.
4. **WARNING!** This product is designed to help people with reduced mobility to walk independently. Misuse of the walker can lead to falls.
5. **WARNING!** DO NOT stand on the walker.
6. **WARNING!** Always keep a safe distance between your rollator and other pedestrians or objects.
7. **WARNING!** Do not attempt to replace the rollator wheels. If they are defective or need to be replaced, please contact the service department.
8. **WARNING!** Make sure the brakes are locked before you sit down and unlock them to start walking.
9. **WARNING!** Make sure the walker is fully unfolded before use.

2. PURPOSE OF THE PRODUCT

- 1. Mobility assistance:** Designed to support people with reduced mobility, improving their balance and allowing them to walk more safely and independently.
- 2. Integrated seat for breaks during the journey:** Incorporates a seat and backrest that allows the user to sit down and rest when needed.
- 3. Use on different surfaces:** Its large and resistant wheels allow it to be moved on different surfaces, making it suitable for the home, shopping centres, parks, etc.

3. BENEFITS AND CONSIDERATIONS

This walker provides numerous benefits for those who need support when walking. These include increased independence, improved balance, and the ability to rest at any time thanks to its built-in seat. In addition, its lightweight, foldable design makes it easy to transport and store, while the storage bag makes it easy to carry personal items in comfort.


However, it is important to take into account some considerations for safe and correct use. The user should ensure that the walker is fully deployed before use, regularly check the condition of the wheels and brakes, and avoid overloading the bag.

4. PRODUCT FEATURES

PRODUCT NAME	Folding walker
MODEL	TRAJANO
MODEL REFERENCE	A-00380/48
MAXIMUM SUPPORTED WEIGHT	299,82 lb
NET PRODUCT WEIGHT	17,85 lb
PRODUCT DIMENSIONS	36,41"-31,49" * 26,37"* 23,62"
COLOUR	Rose gold
MATERIAL	High-quality aluminium alloy, PVC wheels and nylon mesh seat

OTHER CHARACTERISTICS

- Seat size: 17,91"* 9,64"
- Seat to floor distance: 20,86"
- Folded size: 25,19"*10,62"*18,89"
- Size of the basket: 12,59"*14,56" (the backpack has a strap to be used as a cross-body bag)
- Pole holder diameter: 2,16"
- Diameter of wheels: 8"

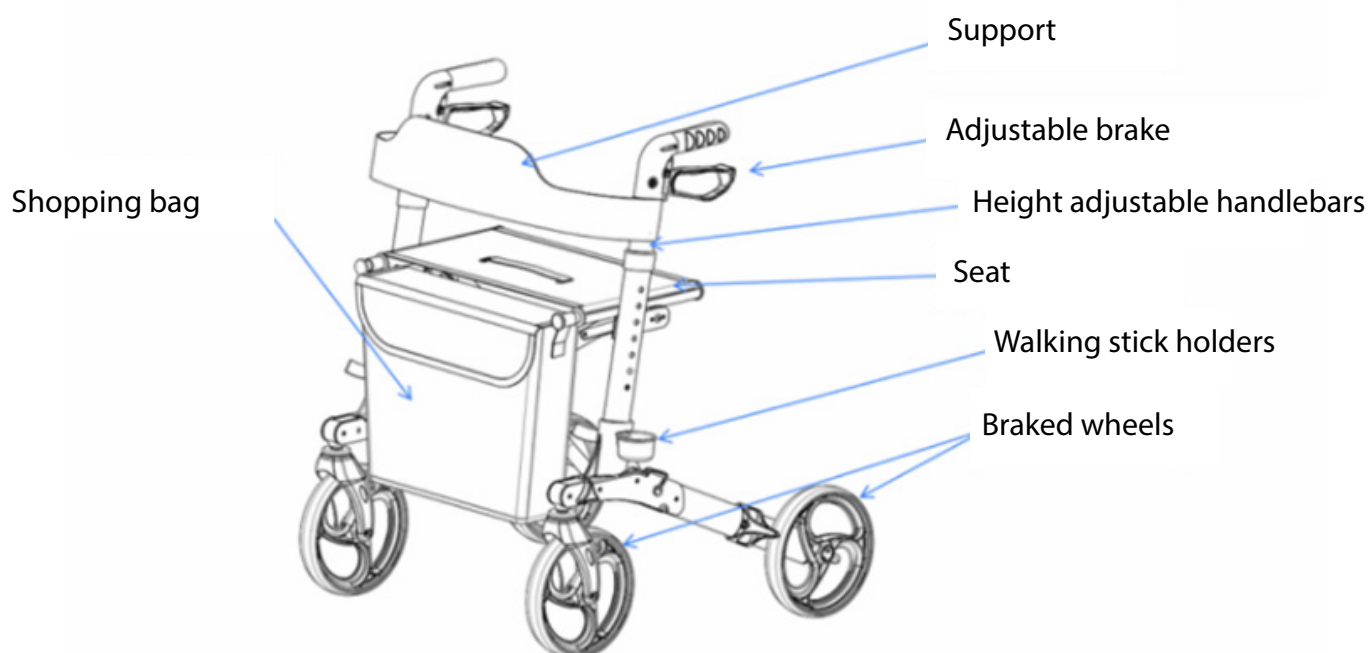5. PARTS AND COMPONENTS**6. INDICATIONS AND CONTRAINDICATIONS****INDICATIONS**

This product is intended for:

1. People with reduced mobility.
2. People recovering from injury.
3. Persons not exceeding the maximum supported weight (299,82 lb).
4. Recommended for people over 59,05" tall.

CONTRAINDICATIONS

This product is not intended for:

1. Dependent patients or patients who are unable to walk with ease.

7. ASSEMBLY INSTRUCTIONS

STEP 1: When you take the walker out of the box, you will find it completely folded as shown in figure 1.

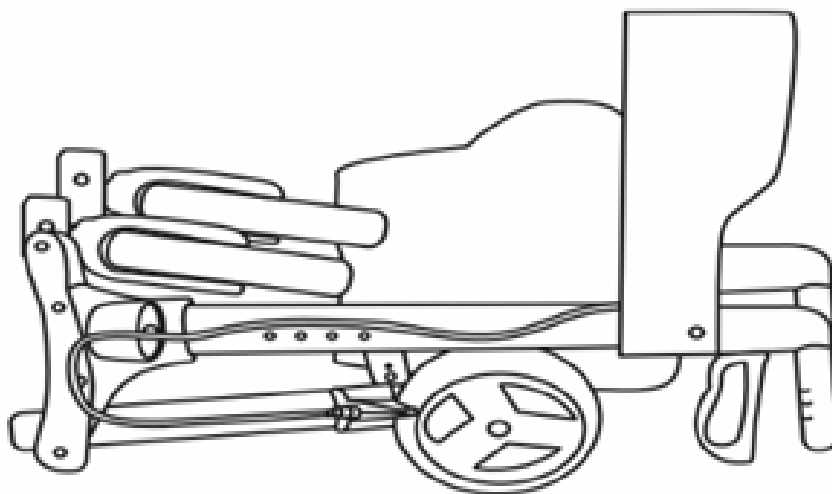
FIGURE 1

STEP 2: Unfold the front wheels and secure them in place with the star-shaped adjusting levers as shown in Figure 2. Make sure the lever is threaded through the hole in the leg and into the frame nut.

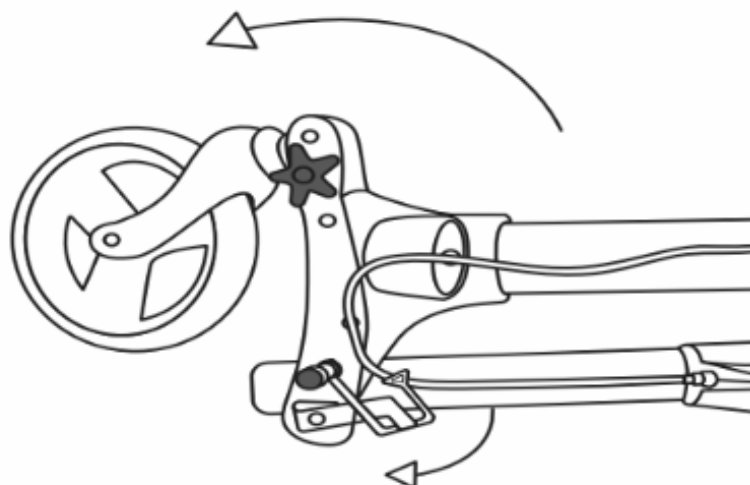
FIGURE 2

STEP 3: Unfold the rear legs as shown in Figure 3 and secure them in place with the star-shaped adjustment knobs. Ensure that the wing nut is threaded through the hole in the leg and into the frame nut. Check that the bolts on the leg knobs of the rear legs are tightened securely.

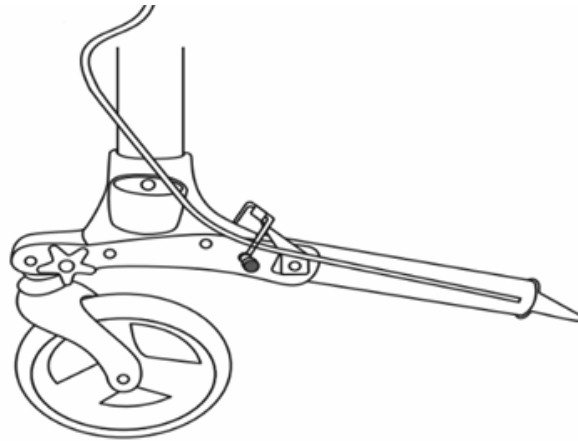
FIGURE 3

STEP 4: To open the walker, push down on both sides of the cross bars as shown in Figure 4. You should feel both sides snap into position.



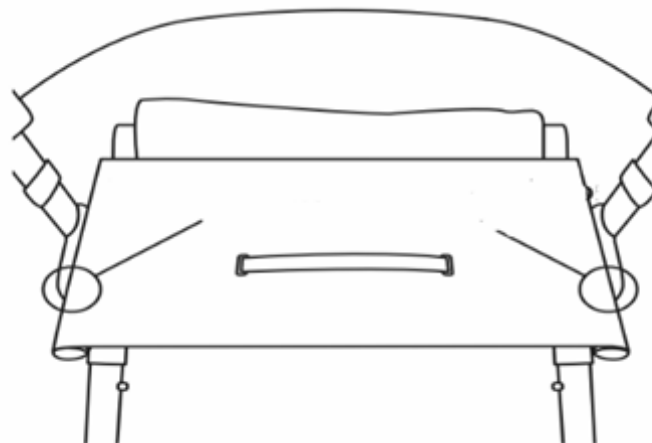
FIGURE 4

APPROPRIATE HEIGHT ADJUSTMENT:


To set the correct height, stand up straight with your arms at your sides. Adjust the handle to wrist height.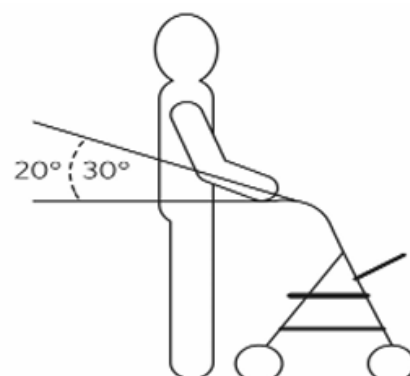

It is important to position the walker correctly. If the handles are positioned too low or too high it can cause strain on the user's shoulders.

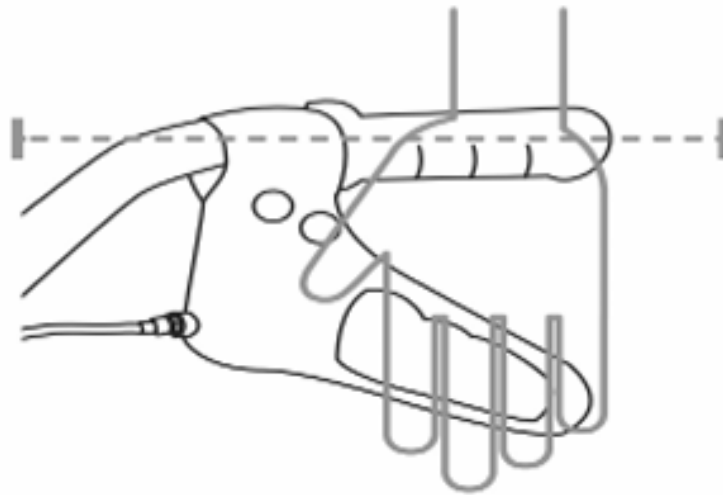
The holes for height adjustment are located on both sides of the walker frame.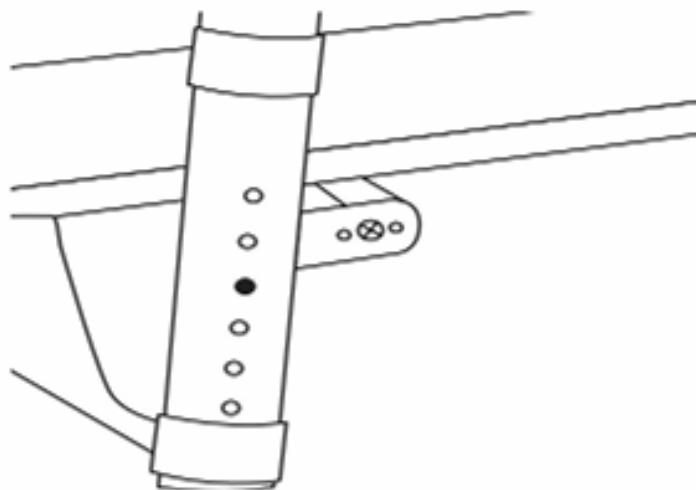


USE OF THE BRAKE:

1. Press the brake handle to brake the walker for specific causes.
2. Push down on the brake handles until they reach the stop to fully lock the rear wheels, in case the user wants to sit down or make a long stop.
3. To release the brake, pull the brake handle lever upwards.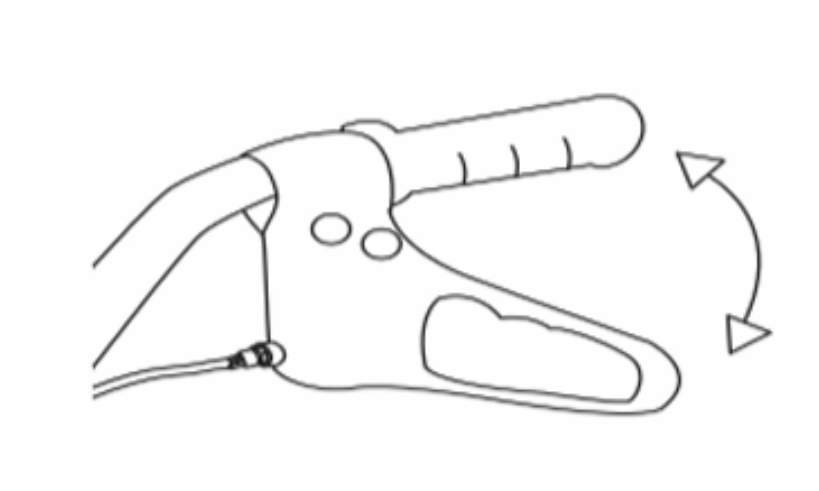

Brake adjustment (in case you notice that the brake is not correctly tensioned):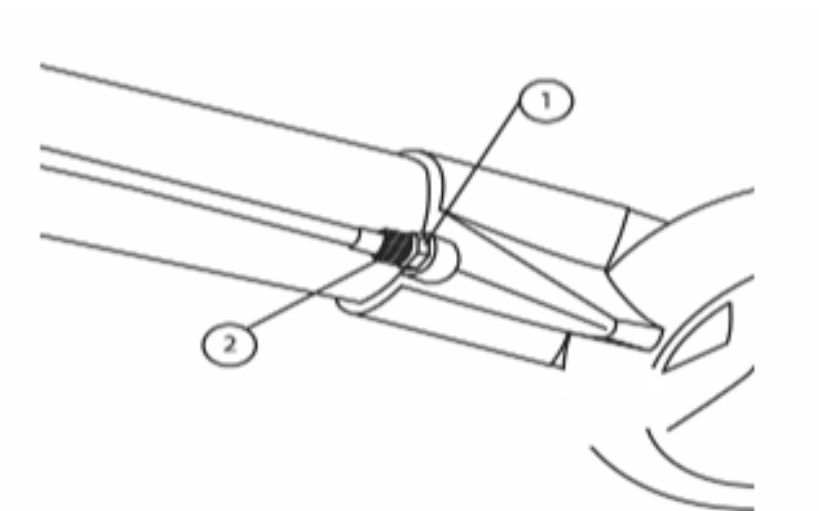

1. Locking nut.


2. Adjusting screw.

Turn the adjusting screw counterclockwise to increase the tension and lock the adjustment by turning the locking nut clockwise.

This process has to be done from time to time to keep the brakes always tightened.

FOLDING OF THE WALKER:

To fold the walker, gently pull the strap on the seat.

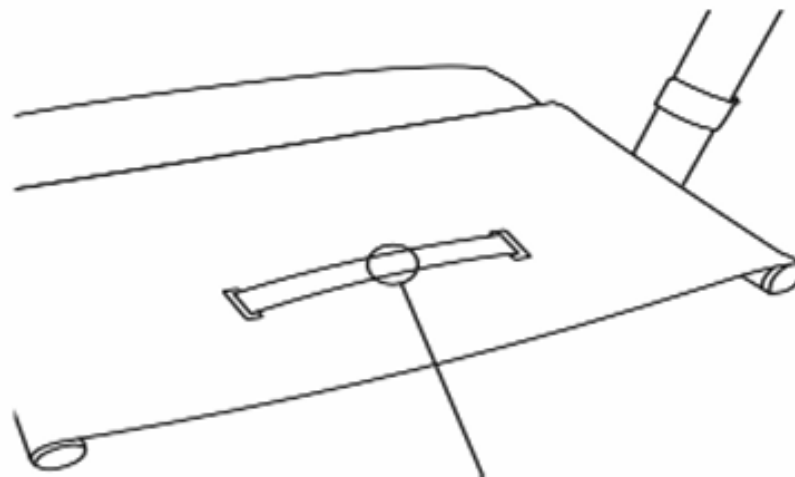8. MAINTENANCE AND CLEANING

Before each use of the walker:

1. Check that the wheels are securely fastened.
2. Check that the brakes are working properly.
3. Brake cables may become loose with age or use of the walker and should be checked regularly.

The walker can be cleaned with a damp cloth.

Do not use scouring pads or scrapers as they may damage the surface coating.

To ensure safety and prolong the service life of the walker, it is recommended to carry out regular inspections and basic maintenance of the most important components:

- 1. Wheels:** Visually inspect the wheels at least once a week. Make sure they turn freely and without unusual noises.
- 2. Brakes:** Check that the brakes respond when the brake handles are applied. The walker should not move when the brakes are applied.
- 3. Seat and backrest:** Ensure that the seat is firm and well fitted, with no tears or sagging.

9. PROBLEM SOLVING

Problem: Front wheels do not turn smoothly.

Cause: Dirt or debris accumulated on the shaft.

Solution: Wipe the wheels with a damp cloth and make sure there is no hair, dust or debris blocking movement.

Problem: Brake does not work properly.

Cause: Brake cable loose or incorrectly adjusted.

Solution: Check the cable tension. If it is loose, tighten the tensioning screw.

10. CONDITIONS OF GUARANTEE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you for choosing Mobiclinic for this TRAJANO folding walker! We appreciate your trust in our products and are committed to always offering you the best quality.

If you have any questions or need additional assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com. We appreciate your comments and suggestions for further improvement.

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XIAMEN NEAQ IMP.& EXP. CO., LTD
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.