MANUAL DE INSTRUCCIONES

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**IMPORTANT**

Keep the instructions for future reference. Read these instructions before assembling and using this product.

Thank you for choosing the non-contact infrared thermometer TO-01! In this manual you will find all the information you need to use and maintain this product correctly and safely.

Before you start, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. And this product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

1. WARNINGS AND PRECAUTIONS

- 1. WARNING!** Portable and mobile frequency communication devices may influence your performance, so you should stay away from these devices.
- 2. WARNING!** The thermometer must not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the thermometer must be observed to verify normal operation in the configuration in which it will be used.
- 3. WARNING!** The thermometer may not be used for transport, such as ambulances (ground and air).

TABLE 1. ELECTROMAGNETIC EMISSIONS

GUIDANCE AND DECLARATION - ELECTROMAGNETIC EMISSIONS	
The thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the thermometer must ensure that it is used in such an environment.	
EMISSIONS TEST	COMPLIANCE
RF emissions CISPR 11	Group 1, Group B

TABLE 2. ELECTROMAGNETIC IMMUNITY

GUIDANCE AND DECLARATION - ELECTROMAGNETIC EMISSIONS		
IMMUNITY TEST	COMPLIANCE	
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV contact ±5 kV air	
Supply frequency (50/60 Hz) Magnetic field IEC 61000-4-8	30 A/m	
RF radiation IEC 61000-4-3	80MHz-2.7GHz	10 V/m
	380MHz-390MHz	27 V/m
	430MHz-470MHz	28 V/m
	704MHz-787MHz	9 V/m
	800MHz-960MHz	28 V/m
	1.7GHz-1.99GHz	28 V/m
	2.4GHz-2.57GHz	28 V/m
	5.1GHz-5.8GHz	9 V/m

TABLE 3. NOT APPLICABLE

Harmonic emissions (IEC 61000-3-2), Voltage flicker emissions (IEC 61000-3-3-3), Electrical fast transients (IEC 61000-4-4), Overvoltages (ICE 61000-4-5), Voltage sags (IEC 61000-4-11), Conducted immunity (IEC 61000-4-6)
Note: These guidelines may not be applicable in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

2. PURPOSE OF THE PRODUCT

The main purpose of an infrared thermometer is to measure temperature at a distance, without the need for physical contact with the object or person whose temperature is to be measured.

This offers advantages such as hygiene, safety, efficiency, precision and convenience.

3. PRODUCT FEATURES

The infrared thermometer is suitable for people of all ages.

The TO-01 device allows you to take your body temperature by receiving infrared energy radiating from your forehead. It is hygienic, accurate and easy to use. The user simply directs the infrared probe to the forehead, presses the measure button and the body temperature is displayed on the LCD screen.

The device is commonly used in hospitals and as an aid to daily living.

It can also be used to measure temperatures from 0°C to 118°C on objects.

The device must not be used in oxygen, air or flammable gas, or nitrogen oxide combinations.

PRODUCT NAME	Non-contact infrared thermometer
MODEL	TO-01
MODEL REFERENCE	X-00001/16
NET PRODUCT WEIGHT	0,28 lb(with batteries)
PRODUCT DIMENSIONS	3,46"* 0,09"*0,33" (length x width x height)
OTHER CHARACTERISTICS	<ul style="list-style-type: none"> • High precision infrared sensors. • Adaptability to ambient temperature. Can be used in complex environments. • Patented sensor designed to ensure more accurate measurement. • Fever alarm function. • Capacity to store 32 temperature readings. • Units of measure: Celsius (°C) or Fahrenheit (°F) • Automatic shutdown and energy saving functions.

4. TECHNICAL DATA

Temperature measurement method		Non-contact
Distance		0,39"-1,18"
Measuring mode	Forehead thermometer mode	32.0°C - 42.0°C (85.9°F - 109.2°F)
	Thermometer mode for object	0°C - 118°C (32°F - 244°F)
Measurement accuracy	Forehead thermometer mode	±0.2°C / 0.4°F
	Thermometer mode for object	±1.0/1.8°F
Resolution	0.1°C/°F	
Operating conditions	10°C~ 40°C (50°F ~ 104°F)(With a relative humidity of up to ≤ 80%)	

SURFACE OBJECT

Storage conditions	-25°C - 55°C / -13°F - 131°F (With relative humidity up to ≤90%)
Batteries	DC3V (2 AA batteries) NOT INCLUDED
Energy consumption	Off: less than or equal to 10mw
	Operating: less than or equal to 30mw
Level indicator	Low battery indication
Storage	Capacity to store 32 temperature readings
Units of measurement	Celsius (°C) or Fahrenheit (°F)
Automatic shutdown	10 seconds

5. PARTS AND COMPONENTS

1. Infrared sensor
2. LCD
3. Backlight on / off button
4. Up button
5. Down button
6. Function key and unit of measure conversation
7. Ignition
8. Battery compartment cover
9. Handle

6. FUNCTIONS OF THE MAIN BUTTONS

1. Function button: Function description.
2. Backlight button: Increased backlighting for use of the device during low-light hours.
3. Option button: Adjustment of parameter settings.
4. Conversion button: Select unit of measurement in the settings.

7. INDICATIONS AND CONTRAINDICATIONS

INDICATIONS

- Routine monitoring of body temperature in adults and children.
- Rapid temperature detection in public places for prevention and control of epidemics (e.g. influenza).
- Fast, non-contact measurement to minimise the risk of contagion.
- Recommended for use in hospitals, clinics, schools, offices and homes.
- Ideal for measuring temperature without discomfort to patients, especially babies and children.
- It can also measure the temperature of objects or surfaces.
- Easy to use with immediate digital readout for frequent monitoring.

CONTRAINDICATIONS

- The results are not a substitute for medical diagnosis.
- Keep out of the reach of children.
- Do not point the device at the eyes.
- Do not change the factory settings.
- Use in a stable temperature environment; allow the device to acclimatise if the environment is changed.
- Wait 10 minutes after measuring temperature extremes before measuring again.
- Do not immerse in liquids.
- Do not use in extreme temperature environments for prolonged periods of time.
- Avoid knocks, falls and do not disassemble the device.
- Do not use in environments with strong electromagnetic interference.
- Do not measure while blow-drying, wet, sweating or wearing cosmetics.
- Do not measure after sport, after washing or within 30 minutes before a meal.
- Maintain a minimum distance of 11,81" from radio frequency communication equipment to avoid interference.

8. CONFIGURATION

The thermometer is provided with 4 types of settings:

- Temperature units.
- Sound switch.
- Temperature deviation.
- Temperature acquisition mode: The mode setting is carried out on the mode change button.

All other settings must be made from the main menu of the thermometer.

The menu is as follows:

MENU	FUNCTIONS	"-"	"+"	INITIAL VALUE	REMARKS
F1	Temperature alarm	Reduce 0.1°C	Increase 0.1°C	37.8°C	Range of 37°C-42.9°C
F2	OffsetTC	Downward migration of 0.1°C	Upward migration of 0.1°C		Range -2°C-2°C
F3	Warning indicator	Close	Open	Open	Press +/- to select the desired function
F4	Measuring mode	Object temperature	Body temperature	Body temperature	Press +/- to select the desired function

TEMPERATURE ALARM SETTINGS F1

Press the "MODE" key, the display will show "F1". From the set alarm, press the "+" button to increase the temperature by 0.1°C or the "-" button to decrease the temperature by 0.1°C. To finish, press the "MODE" key to confirm.

TEMPERATURE COMPENSATION SETTINGS F2

Press the "MODE" key, the display will show "F1", press "MODE" again to display "F2". Then, set the temperature compensation. To finish, press the "MODE" key again to save the update you have made.

F3 WARNING SETTINGS

Press "MODE" key, the display will show "F1", press twice more to show "F3", press "MODE" again to set the tone, the display will show "ON". Press "+" or "-" to make the display show "OFF" and disable the prompts, finally press "MODE" again to save the update you have made.

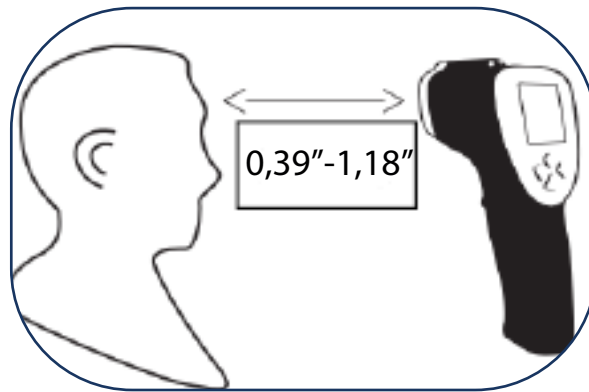
MEASUREMENT MODE F4

Press "MODE" key, the display will show "F1", press "MODE" key three times to display "F4", press "MODE" again to choose the temperature taking mode, then the display will show "surf", press "+" or "-" and the display will show "body". again to choose the temperature taking mode, then the display will show "surf", press "+" or "-" and the display will show "body". To save the selected temperature mode press "MODE".

9. TAKE THE TEMPERATURE

TAKING BODY TEMPERATURE

1. Press the measurement key to turn on the device, then the display will show the backlight and all symbols will appear on the display. The device will beep twice when the self-test is finished and the display will show the last value stored in the memory and then enter the measuring status.
2. Ensure that the measurement mode is set to body temperature mode.
3. Align the thermometer probe with the forehead, the distance should be about 0,39"-1,18". Press the measuring key. The temperature will then appear on the display when a signal sounds and 0.5 seconds elapse. If the temperature exceeds the limit (the default value is 37.8°C) an alarm will sound.
4. When the measurement is finished, OFF will appear on the display and it will turn off after 30 seconds.

**NOTES:**

1. Keep the sensor and internal cavity clean before taking the temperature.
2. Use the thermometer in a stable temperature environment. If ambient temperature changes are high, wait about 10 minutes before taking the temperature.
3. Do not measure body temperature after measuring extremely high or low temperatures, wait 10 minutes before measuring.

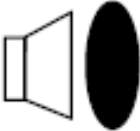
TAKING THE TEMPERATURE OF THE OBJECT

1. Press the measurement key to switch on the device and activate the measurement status.
2. Make sure that in the upper display the body temperature mode is not set, but the surface temperature mode is set.
3. Align the thermometer sensor with the centre of the object, the distance should be 5 to 15cm. Press the measure key, the temperature will be shown on the display after a beep, in 0.5 seconds.
4. The thermometer shall automatically switch off after 10 seconds of non-operation.

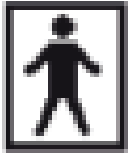
10. BATTERY CHANGE

- Battery life is approximately 3000 readings in one year.
- When the thermometer indicates low battery, it is recommended to replace the batteries.
- Open the cover and remove the used batteries.
- Insert new batteries and be careful with the position of each battery.
- Tip: When not using the thermometer for a while (about 3 months), please remove the batteries.

11. EXPLANATION OF SYMBOLS

DEFINITION OF SYMBOLS	SYMBOLS	DESCRIPTION
Sound		Sound warning
		Beep, not a sound
Measuring mode	Body temperature	Body temperature mode
	Surface temperature	Object temperature mode
Unit of measurement	°C	Degrees Celsius
	°F	Degrees Fahrenheit
Numeric	888.8	Recorded temperature value
Own light		Display backlight
Storage	LOG	32 latest temperature readings

12. ICONS

	Recycling
	Warning
	Flammable and explosive
	Read the instruction manual before use
	Class BF medical device

13. MAINTENANCE

- Keep the sensors and the internal cavity clean. Otherwise, measurements may give inaccurate results.
- This device only requires routine cleaning, at times that depend on the user's needs.
- To clean the surface, use a clean, soft cloth or a cotton swab dipped in sanitary alcohol or water to remove dirt.
- To clean the sensor and internal cavity, use a clean, soft cloth or cotton swab with a little medical alcohol to gently wipe the sensor and internal cavity. Do not use the thermometer until it is completely dry.
- The process of use does not require any maintenance.

14. PROBLEM SOLVING

DIAGNOSIS	SOLUTIONS
The display shows "Lo" or "Hi".	<ol style="list-style-type: none"> 1. Check the object on which the temperature is to be taken; note that the accuracy of the forehead temperature measurement is not guaranteed if the device is placed on hair, water, perspiration or cosmetics. 2. Check the temperature deviation setting. 3. In case the temperature change is too large or in case you use the thermometer with an object that is at low temperature after using it at high temperature, it is recommended to use it after 10 minutes to achieve a good thermal equilibrium. 4. Check the recommended distance, which should be 1,96" to 5,90".
Buttons do not respond	<ol style="list-style-type: none"> 1. Remove the batteries and put them back in. 2. Check if the device configuration is in progress.
Display does not work or works incorrectly	Remove the batteries and put them back in again.
Thermometer does not turn on	Check that the batteries are inserted correctly and that they are working properly, remove and reinsert them.

15. EMC INFORMATION - GUIDANCE AND MANUFACTURER'S DECLARATION

1. The thermometer complies with the electromagnetic compatibility requirement of IEC60601-1-2.
2. The user needs to install and use in accordance with the EMC information enclosed with it.
3. Portable and mobile RF communication devices may affect performance and the thermometer should be kept away from them during use.
4. Guidance information and the manufacturer's declaration can be found in the appendix.

16. CONDITIONS OF GUARANTEE

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for having trusted Mobiclinic for this non-contact infrared thermometer TO-01! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or need further assistance, please do not hesitate to contact us through our customer service at clientes@mobiclinic.com . We appreciate your comments and suggestions for further improvement.

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SHENZHEN BRAV ELECTRONIC TECHNOLOGIES CO., LTD
GRUPO R. QUERALTÓ S.A.