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## IMPORTANT

Keep these instructions for future reference. Read these instructions before assembling and using this product. Adult assembly is required.

Thank you for choosing the SAN FERMÍN bath chair! In this instruction manual, you will find all the information you need to assemble, use, and maintain the chair safely.

Before you begin, we recommend that you read this manual carefully to ensure optimal and safe use of the product.

As a guarantee of the quality of our products, the manufacturer has a management system certified under ISO 13485. This product complies with the requirements of Regulation (EU) 2017/745 on medical devices.

Notice to the user and/or patient: any serious incident that has occurred in relation to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 1. WARNINGS AND PRECAUTIONS

1. **WARNING!** Do not use the bath chair if it is damaged, has loose parts or shows signs of wear that could compromise its stability.
2. **WARNING!** Ensure that all parts are correctly assembled and adjusted before each use.
3. **WARNING!** Do not exceed the maximum weight recommended by the manufacturer.
4. **WARNING!** Only use the chair on stable, level surfaces. Do not place it on sloping, slippery or unstable floors.
5. **WARNING!** Do not use the chair as a step or support to stand up if it is not designed for that purpose.
6. **WARNING!** Avoid contact with harsh chemicals that could damage the chair's materials.
7. **WARNING!** Do not leave persons with reduced mobility unattended while using the chair, especially if they require assistance to sit down or stand up.
8. **WARNING!** Regularly check the condition of the chair and perform maintenance according to the manufacturer's recommendations.

## 2. PURPOSE OF THE PRODUCT

This shower chair has been designed to provide safety, stability, and comfort for people with reduced mobility, those in recovery, or with disabilities. Its sturdy and ergonomic structure offers firm support, ensuring safe use during personal hygiene.

### 3. BENEFITS AND CONSIDERATIONS

#### BENEFITS:

- Makes bathing easier for people who have difficulty standing for long periods of time.
- It has adjustable legs to adapt to different heights and user needs, ensuring greater comfort.
- Manufactured with high-quality materials that withstand frequent use without deteriorating.


#### CONSIDERATIONS:

- Ensure that the chair is placed on a firm, level surface before use.
- Do not exceed the maximum weight recommended by the manufacturer, 220,46 lb.
- Ensure that the adjustable legs are securely fastened and locked before sitting down.
- Regularly check the frame, screws, and legs to ensure they are in good condition and securely fastened.

### 4. PRODUCT FEATURES

<b>PRODUCT NAME</b>	Bath chair
<b>MODEL</b>	SAN FERMÍN
<b>MODEL REFERENCE</b>	A-00465/01   A-00465/01-NG
<b>COLOURS</b>	A-00465/01 (Blanco) A-00465/01-NG (Black)
<b>MAXIMUM WEIGHT SUPPORTED</b>	220,46 lb
<b>NET WEIGHT OF THE PRODUCT</b>	3,96 lb
<b>PRODUCT DIMENSIONS</b>	16,14"*20,07"*16,92"-22,83"
<b>MATERIALS</b>	Aluminium, steel and PVC
<b>OTHER FEATURES</b>	<ul style="list-style-type: none"> <li>• Height adjustable from 16,92" to 22,83"</li> <li>• Seat dimensions: 20,07"*12,20"</li> <li>• Non-slip tips</li> <li>• Tip diameter: 1,10"</li> </ul>

## 5. PARTS AND COMPONENTS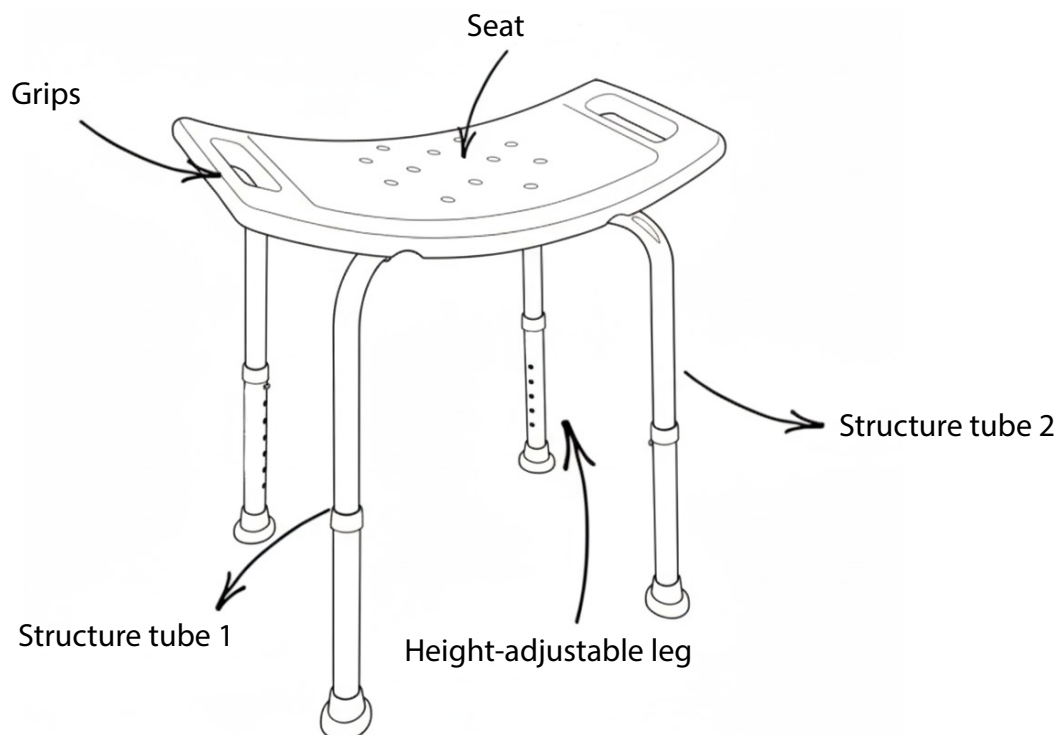

## 6. INDICATIONS AND CONTRAINDICATIONS

### INDICATIONS

This high chair is recommended for:

- People with weakness or fatigue in their lower limbs: for example, those with insufficient strength in their legs due to ageing, post-operative recovery or chronic illnesses, making it difficult for them to stand for long periods of time while showering.
- People with slightly impaired balance.
- People with joint pain or limited mobility.

### CONTRAINDICATIONS


This shower chair is not recommended for:

- Individuals weighing over 220,46 lb, as their structure could be damaged and increase the risk of falls.
- Individuals unable to maintain their balance when sitting independently.
- Individuals with cognitive impairment or altered consciousness.

## 7. ASSEMBLY INSTRUCTIONS


### STEP 1:

- Place the seat on a flat, stable surface.
- Align structural tube 1 following the X-shaped seat path.
- Please note that the structural tube (1) is the one with the most flattened centre.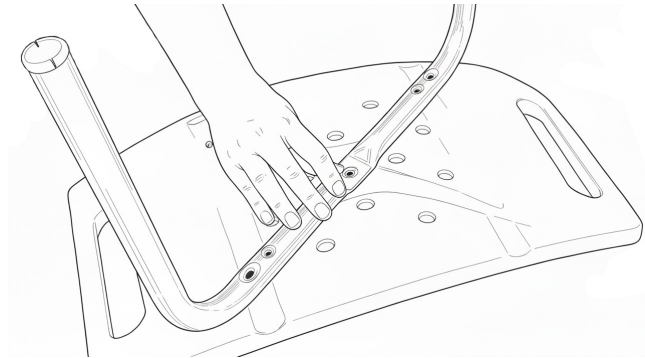


### STEP 2:

- Align structural tube B in the opposite direction of the seat to complete the "X".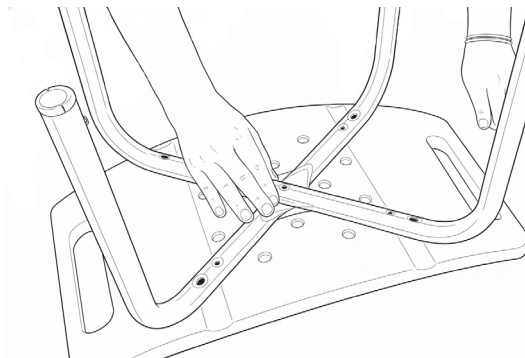

### STEP 3:


- Secure the two tubes of the structure. To do this, place the screw and washer in the hole connecting the two tubes of the structure.
- Repeat the process with the holes located on the tubes of the structure. You should only secure the holes located at the ends.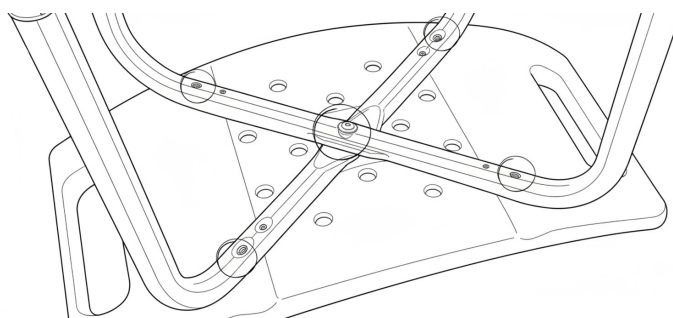

## INSTALLATION OF ADJUSTABLE LEGS

1. Place the adjustable legs at the ends of the tubes of the structure.
2. Press the silver button and insert the adjustable leg until you hear a "click", indicating that it is correctly fixed at the desired height.
3. Repeat the process with all 4 legs.

Once assembled, it should look like this:

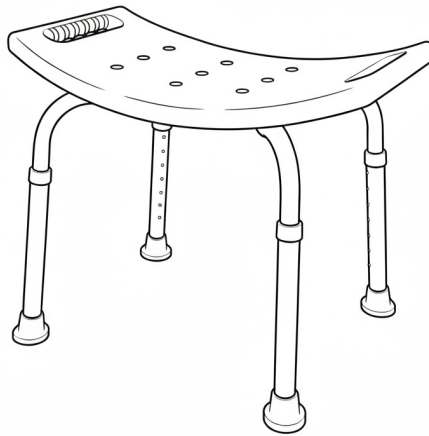## 8. INSTRUCTIONS FOR USE

- Place the chair on a stable, non-slip surface, ensuring that all legs are properly supported.
- Before sitting down, check that the chair is properly assembled and adjusted, ensuring that the screws and parts are secure.
- The chair is height adjustable, adjust it for your safety, ensuring comfort and security.
- Ensure that the user is correctly seated and balanced before starting the bath.
- After each use, dry the chair thoroughly to prevent moisture build-up and prolong its durability.

## 9. MAINTENANCE AND CLEANING

- Clean the product only with soap and water; do not use any chemical substances.
- Protect it from adverse weather conditions.
- Periodically check that all parts are securely fastened and in good condition.
- Ensure that the adjustment mechanisms are functioning correctly and are not blocked.

## 10. PROBLEM SOLVING

**Q:** The bath chair wobbles or is unstable.

**S:** Ensure that all legs are securely fastened and that the surface on which it is placed is flat.

**Q:** The chair's non-slip feet slip in the shower or bath.

**S:** Check that the non-slip rubber tips are in good condition and properly fitted. If they are worn, replace them. Their diameter is 1,10".

**Q:** The height of the chair does not adjust correctly.

**S:** Ensure that you unlock the adjustment mechanisms before attempting to change the height and that you reattach them correctly.

**Q:** The chair leaves marks or residue in the bath.

**S:** Clean the adjustable legs properly and ensure that the surface is free of soap or accumulated dirt.

**Q:** The chair seat feels unstable or moves.

**S:** Check the screws and tighten them if necessary to ensure secure mounting.

## 11. WARRANTY CONDITIONS

This product is covered by a 36-month warranty against manufacturing defects.

Thank you very much for choosing Mobiclinic for this SAN FERMÍN shower chair! We value your trust in our products and are committed to always offering you the best quality.

If you have any questions or require further assistance, please do not hesitate to contact us via our customer service at [clientes@mobiclinic.com](mailto:clientes@mobiclinic.com). We appreciate your comments and suggestions to help us continue to improve.

Thank you for being part of Mobiclinic!

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CO., LTD**
**SUNGO EUROPE B.V.**
**GRUPO R. QUERALTÓ S.A.**